


# eSight 3 EXTENDED BATTERY PACK QUICK START

## Powering Your eSight 3

**NOTE:** Charge the extended battery before first use. See “Charging Your External Battery”.

- 1 Fit the teeth on the battery holster cradle into the slots on the side of the eSight 3 controller.
- 2 Plug the small end of the thick USB cable into the USB connection on your controller.
- 3 Clip the holster to your belt or wear the holster over your shoulder.

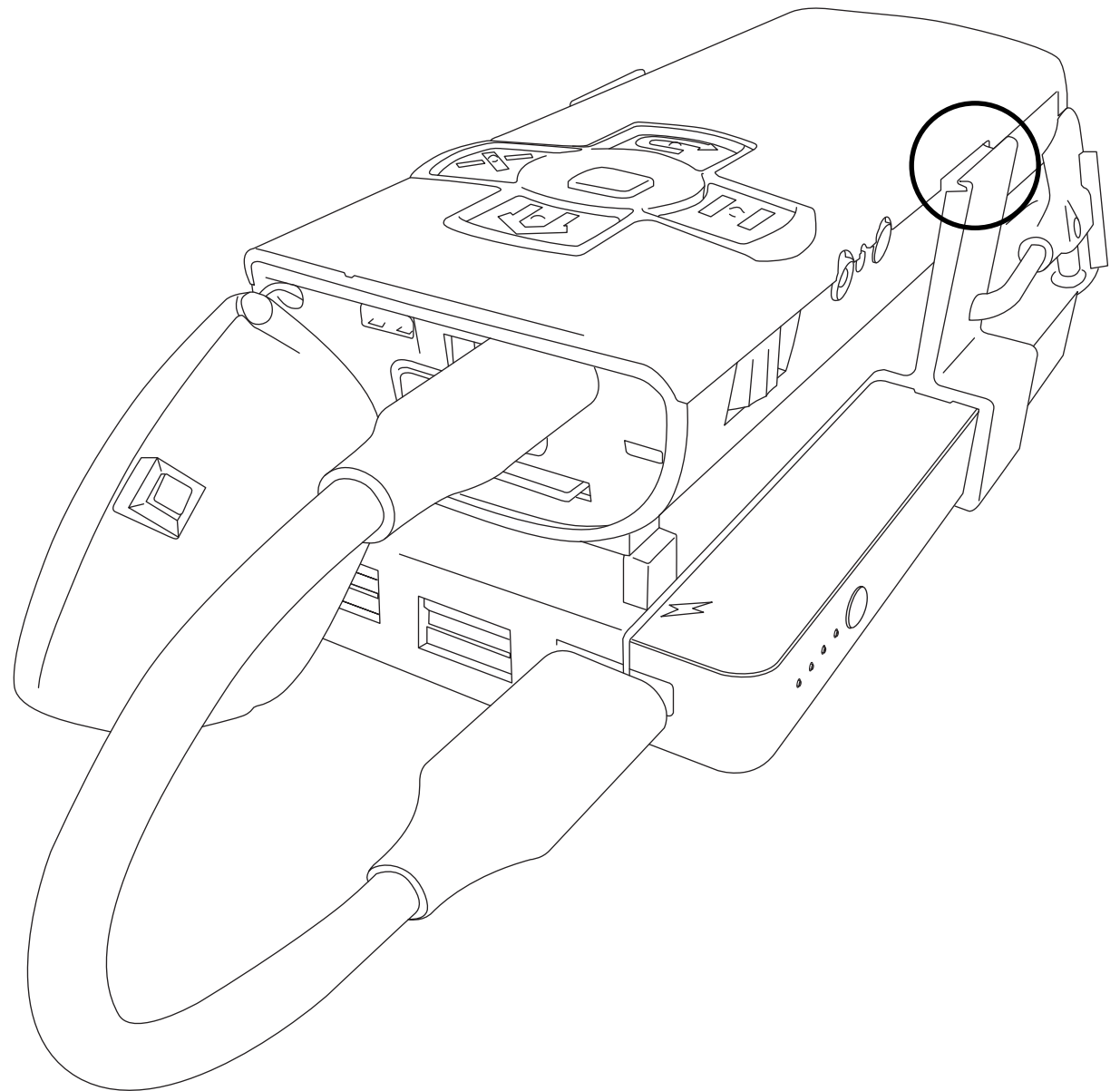

---


## Charging Your External Battery

- 1 Disconnect your eSight 3.
- 2 Using the thin USB cable, plug the small end into the center connection on the external battery.
- 3 Plug the other end into your eSight 3 wall adapter.
- 4 Plug the adapter into a wall outlet.

**TIP:** Push the Status Button on the battery for the charge level.

**Four Lights = Fully Charged.**

---

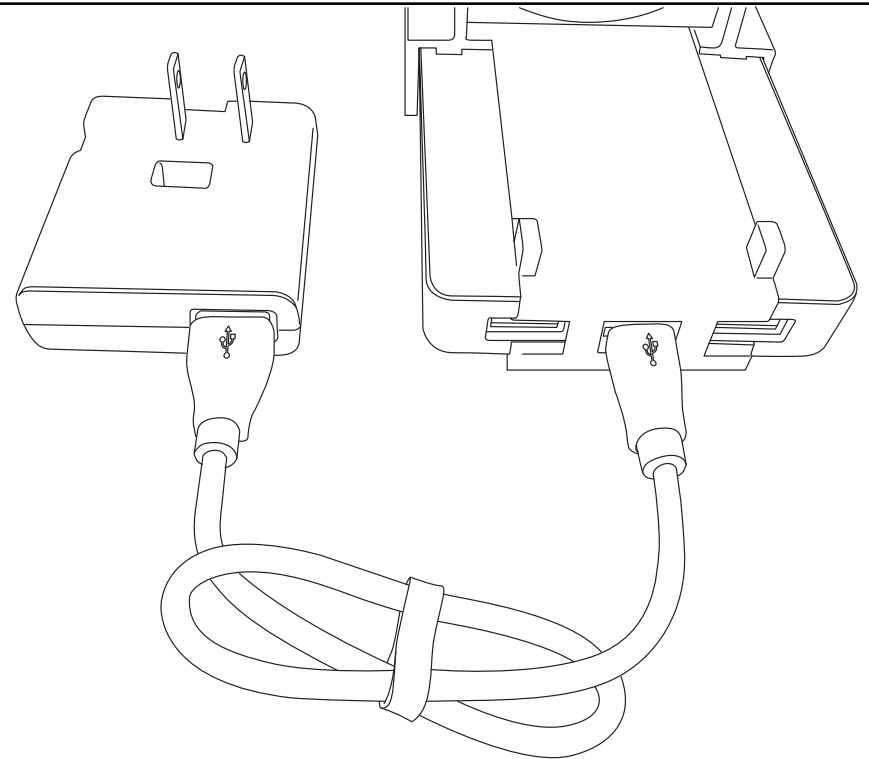

## Charging Your eSight 3 and External Battery

- 1 Power your eSight 3. See page 1 for details.
- 2 Follow steps 2 to 4 of "Charging Your External Battery".

