



What's New?

LOW VISION SOFTWARE, SEPTEMBER 2018

CONTENTS

Recent Improvements.....	1
Version 3.1.27	2
OCR Text to Speech Reader Enhancements	2
Narration Enhancement.....	2
Update Instructions	3
Update Using an SD Card	3
Update Over WiFi	5

RECENT IMPROVEMENTS

The eSight Low Vision Software has some new updates! We hope you enjoy the latest improvements and features.

Version 3.1.27

OCR Text to Speech Reader Enhancements

- OCR works with English, French, Italian, Spanish, and German text. Go to **TOOLS > OCR LANGUAGE** to set the language of the text you want to interpret.
- Push the trackpad or the Photo Button to capture the image of the text.

Narration Enhancement

Narration is now available in English, French, Italian, Spanish and German.

UPDATE INSTRUCTIONS

You can update eSight 3 using either the SD Card Update method or the WiFi Update method.



Update Using an SD Card



Your battery must be at 60% charged (Unplugged) or 20% charged (Charging) to update eSight 3.

1. Ensure eSight 3 is plugged into power and well charged.
2. Put eSight 3 into Airplane Mode. See “Turn on/off Airplane Mode” in the Software Quick Reference.
3. Turn off eSight 3.
4. If necessary, remove the installed SD card by pushing in and releasing the SD card.

For the location of the SD card slot, see the diagram on page 15 of the eSight 3 User Manual.

5. Insert the v3.1.27 SD card into the controller.
6. Simultaneously push the **SOUTH**  and **WEST**  buttons on your controller.

If there is an update, the LED flashes red and green.

When the update is done, the LED shows the status of the battery: Solid Green (Battery Fully Charged) or Solid Yellow (Battery More Than 5% Charged).

7. Turn on eSight 3.
8. Wait five minutes to ensure the SD card is ready for use.
9. Push the trackpad to open the eSight Menu.
10. Go to **SETUP > SYSTEM > UPDATE > SD**.
11. To confirm that you want to upgrade the software on your eyewear, select the **YES** option.

Your eyewear displays the “Checking for System Update” message.

Then, your eyewear restarts and installs the update. The update takes several minutes to install.

Finally, once the update installs, your eyewear restarts again.

12. In the **ABOUT** menu, verify that the following versions display: V3.1.27 CV3.1.11.


If not, contact the Customer Support representative for your region.


13. Remove the v3.1.27 SD card from your controller, and keep it in a safe place.

14. Reinsert your initial SD card.

The update is complete when you can use your eyewear.

Update Over WiFi

 Your battery must be at 60% charged (Unplugged) or 20% charged (Charging) to update eSight 3.

 For this procedure, you require an SD card.

1. Ensure eSight 3 is plugged into power and well charged.
2. If necessary, insert an SD card into the controller.



For the location of the SD card slot, see the diagram on page 15 of the eSight 3 User Manual.

3. Turn on eSight 3.
4. Wait five minutes to ensure the SD card is ready for use.
5. If necessary, turn off Airplane Mode. See “Turn on/off Airplane Mode” in the Software Quick Reference.
6. Connect to a wireless network. See “Connect to a Saved Wi-Fi Network” in the Software Quick Reference.
7. Push the trackpad to open the eSight Menu.
8. Go to **SETUP > SYSTEM > UPDATE > WEB**.
9. To confirm that you want to upgrade the software on your eyewear, select the **YES** option.

The “Checking for System Updates” message appears. If there is an update, the “Downloading Update” message appears.

Then, your eyewear restarts and installs the update. The update takes several minutes to install.

Finally, once the update installs, your eyewear restarts again.

10. When you see the camera video, simultaneously push the **SOUTH**  and **WEST**  buttons on your controller.

If there is an update, the LED flashes red and green.

When the update is done, the LED shows the status of the battery: Solid Green (Battery Fully Charged) or Solid Yellow (Battery More Than 5% Charged).

11. Turn on eSight 3.

12. In the **ABOUT** menu, verify that the follow versions display:
V3.1.27 CV3.1.11.

If not, contact the Customer Support representative for your region.

The update is complete when you can use your eyewear.

IF YOU'RE HAVING TROUBLE INSTALLING YOUR UPDATE

If you encounter any problems installing this update, please leave the controller turned on and contact the Customer Support representative for your region. Consult your user documentation for details.

4000409 Rev. J © Copyright 2018, eSight Corporation. All Rights Reserved.