



# eStreamer Guide

## for Android™


### How do I...?

- Start eStreamer ..... 1
- Connect the Tablet to eSight 3 using WiFi Direct ..... 2
- Connect the Tablet to eSight 3 using Bluetooth ..... 3
- Connect to a Paired Tablet using Bluetooth ..... 3
- View eSight 3 Camera Video on the Tablet ..... 4
- Control eSight 3 from the Tablet ..... 4
- Stream the Tablet Screen to eSight 3 ..... 5
- Stream the Tablet Camera Video to eSight 3 ..... 5
- Turn On/Off the eSight 3 Light ..... 5
- Disconnect eStreamer from eSight 3 ..... 6
- Block eStreamer Connections to eSight 3 ..... 6



eStreamer may not work with all Android devices. Consult your eSight Ambassador for guidance on compatible devices prior to purchasing a device for use with eStreamer.

### Start eStreamer


- On the Home screen, tap  eStreamer.

# Connect the Tablet to eSight 3 using WiFi Direct




If there's more than one eSight 3 near you, you'll need to know the serial number of your eyewear.


From the eSight Menu, go to the **ABOUT** menu.

1. Turn on eSight 3.
2. If necessary, turn off Airplane Mode (No Check Mark).
3. If necessary, set the WiFi connection on your eyewear to WiFi Direct (**SETUP > SYSTEM > CONNECT > WIFI > WIFI DIRECT**).
4. In the eSight Menu, go to **SETUP > SYSTEM > CONNECT > ESTREAMER > ALLOW** (Check Mark) to permit eStreamer use with your eyewear.
5. Start eStreamer.
6. If you've recently connected your eyewear to eStreamer using Bluetooth, Tap  **Settings** and select **Use Bluetooth** (No Check Mark); otherwise, skip this step.
7. In eStreamer, tap **Tap to Connect**.
8. From the list, tap the eyewear to which you want to connect.
9. When you see the **ALLOW ESTREAMER?** message in eSight 3, select **YES** to accept the eStreamer connection for the current session or **REMEMBER** to accept all eStreamer connections made from the tablet.

## Connect the Tablet to eSight 3 using Bluetooth

1. Turn on eSight 3.
2. If necessary, turn off Airplane Mode (No Check Mark).
3. In the eSight Menu, go to **SETUP > SYSTEM > CONNECT > ESTREAMER > ALLOW** (Check Mark) to permit eStreamer use with your eyewear.
4. If necessary, turn on the Bluetooth® connection (**SETUP > SYSTEM > CONNECT > BLUETOOTH > BLUETOOTH** (Check Mark)).
5. Select **PAIR NEW DEVICE**.
6. Start eStreamer.
7. Tap  **Settings**.
8. If necessary, tap **Use Bluetooth** (Check Mark).
9. In eStreamer, tap **Tap to Connect**.
10. From the list, tap the eyewear to which you want to connect.
11. Tap **Pair**.

## Connect to a Paired Tablet using Bluetooth


1. Turn on eSight 3, and check the following:
  - **AIRPLANE MODE** is OFF.
  - **ESTREAMER > ALLOW** is ON.
  - **BLUETOOTH** is ON.
2. Start eStreamer, and check  **Settings > Use Bluetooth** is ON.
3. Tap **Tap to Connect**.
4. Tap the eyewear to which you want to connect.

# View eSight 3 Camera Video on the Tablet

1. Start eStreamer.
2. Connect your tablet to eSight 3.
3. Tap **Stream In**.

## Control eSight 3 from the Tablet

1. View the eSight 3 camera video on the tablet.
2. Tap the tablet screen to display the remote control panel.
3. Tap the on-screen button.

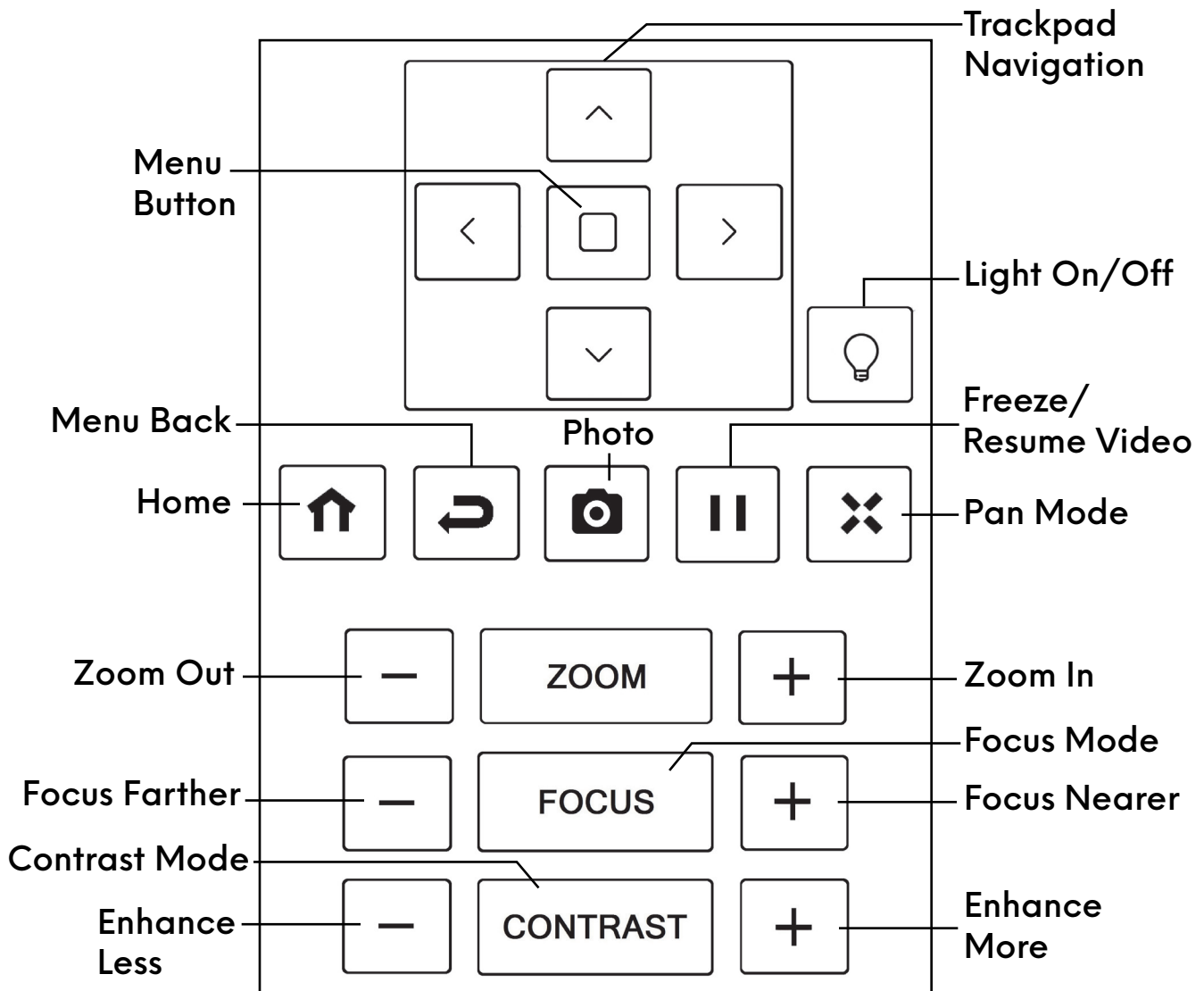

## Stream the Tablet Screen to eSight 3



Google Play will not play videos while you are streaming your tablet screen to eSight 3. This protects copyrighted content from being retransmitted.


If you want to watch a movie on the headset, connect your tablet to eSight 3 using the HDMI cable. From the eSight Menu, go to **VIEW > HDMI**. For more information, see “Connect HDMI Video Source to eSight 3” in the Software Quick Reference.

1. Start eStreamer.
2. Connect your tablet to eSight 3.
3. Tap **Share Screen**.
4. Click **Start Now** to acknowledge the “eStreamer will start capturing everything that’s displayed on your screen.” message.

## Stream the Tablet Camera Video to eSight 3

1. Start eStreamer.
2. Connect your tablet to eSight 3.
3. Tap **Share Camera**.

## Turn On/Off the eSight 3 Light

1. Start eStreamer.
2. Connect your tablet to eSight 3.
3. Tap  **Settings**.
4. Tap **Light ON** or **Light OFF**.



Once you’ve established the initial connection between your eyewear and your tablet, it’s super easy to switch quickly between sharing the tablet camera or screen or viewing eSight 3 camera video — just tap **Share Camera**, **Share Screen**, or **Stream In**.

## Disconnect eStreamer from eSight 3

1. On eSight 3, open the eSight menu.
2. Go to **SETUP > SYSTEM > CONNECT > ESTREAMER > DISCONNECT ALL.**

## Block eStreamer Connections to eSight 3

1. On eSight 3, open the eSight menu.
2. Go to **SETUP > SYSTEM > CONNECT > ESTREAMER > ALLOW** (No Checkmark).

“eSight 3” and “eSight” and associated images and symbols are trademarks of eSight Corporation. “eSight” is a registered trademark in Canada and the U.S. and may be pending or registered in other countries. Various U.S. and other patents pending. All other trademarks, registered trademarks, product names, and company names and/or logos cited herein, if any, are the property of their respective holders.

4000376 Rev. E © Copyright 2018, eSight Corporation.

