



eStreamer Guide

for iOS

How do I...?

Start eStreamer	1
Connect the iPhone to eSight 3 for the First Time	2
Reconnect the iPhone to eSight 3	3
View eSight 3 Camera Video on the iPhone	3
Control eSight 3 from the iPhone (With Video)	4
Use Your iPhone as a Remote Control for eSight 3 (Without Video)	4
Stop Receiving Video	5
Resume Receiving Video	5
Disconnect eStreamer from eSight 3	5
Block eStreamer Access to eSight 3	5



eStreamer may not work with all Apple® devices. Consult your eSight Ambassador for guidance on compatible devices prior to purchasing a device for use with eStreamer.

How to Start eStreamer

- On the Home screen, tap the eStreamer icon .

How to Connect the iPhone to eSight 3 for the First Time



Before you begin, you'll need to know the serial number of your eyewear. From the eSight Menu, go to the **ABOUT** menu.

1. Turn on your eSight 3.
2. If necessary, turn off Airplane Mode (No Check Mark).
WIFI DIRECT is the normal setting for wireless networking with your eyewear. If you've recently connected your eyewear to a wireless access point, such as a router, you must change your WiFi connection in your eyewear from WIFI ROUTER to WIFI DIRECT.
3. If necessary, set the WiFi connection on your eyewear to WiFi Direct (**SETUP > SYSTEM > CONNECT > WIFI > WIFI DIRECT**).
4. In the eSight Menu, go to **SETUP > SYSTEM > CONNECT > ESTREAMER > ALLOW** (Check Mark) to permit eStreamer use with your eyewear.
5. Go to your iPhone Wi-Fi Settings.
6. From the list of available networks, tap the eyewear to which you want to connect.
7. Type **123456789** for the password, and then tap **Join**.
8. Start eStreamer.
9. When you see the **ALLOW ESTREAMER?** message in your eSight 3, push the trackpad to select the **YES** option and accept the eStreamer connection.

How to Reconnect the iPhone to eSight 3

1. Turn on your eSight 3
2. Go to your iPhone Wi-Fi settings.
3. From the list of available networks, tap the eyewear to which you want to connect.
4. Start eStreamer.
5. When you see the **ALLOW ESTREAMER?** message in your eSight 3, push the trackpad to select the **YES** option and accept the eStreamer connection.



If eStreamer doesn't connect to eSight 3, do this:


- i In the eStreamer main menu, go to **CONNECT**.
- ii From **Available Devices** list, tap the eyewear to which you want to connect.
- iii When you see the **ALLOW ESTREAMER?** message in your eSight 3, push the trackpad to select the **YES** option.

How to View the eSight 3 Camera Video on the iPhone

1. Start eStreamer.
2. Connect your iPhone to your eSight 3.
3. In the eStreamer main menu, tap **Stream from your eSight**.

How to Control the eSight 3 from the iPhone (With Video)

1. View the eSight 3 camera video on the iPhone.
2. Tap the camera video to display the remote control panel.
3. Tap to press an on-screen button.

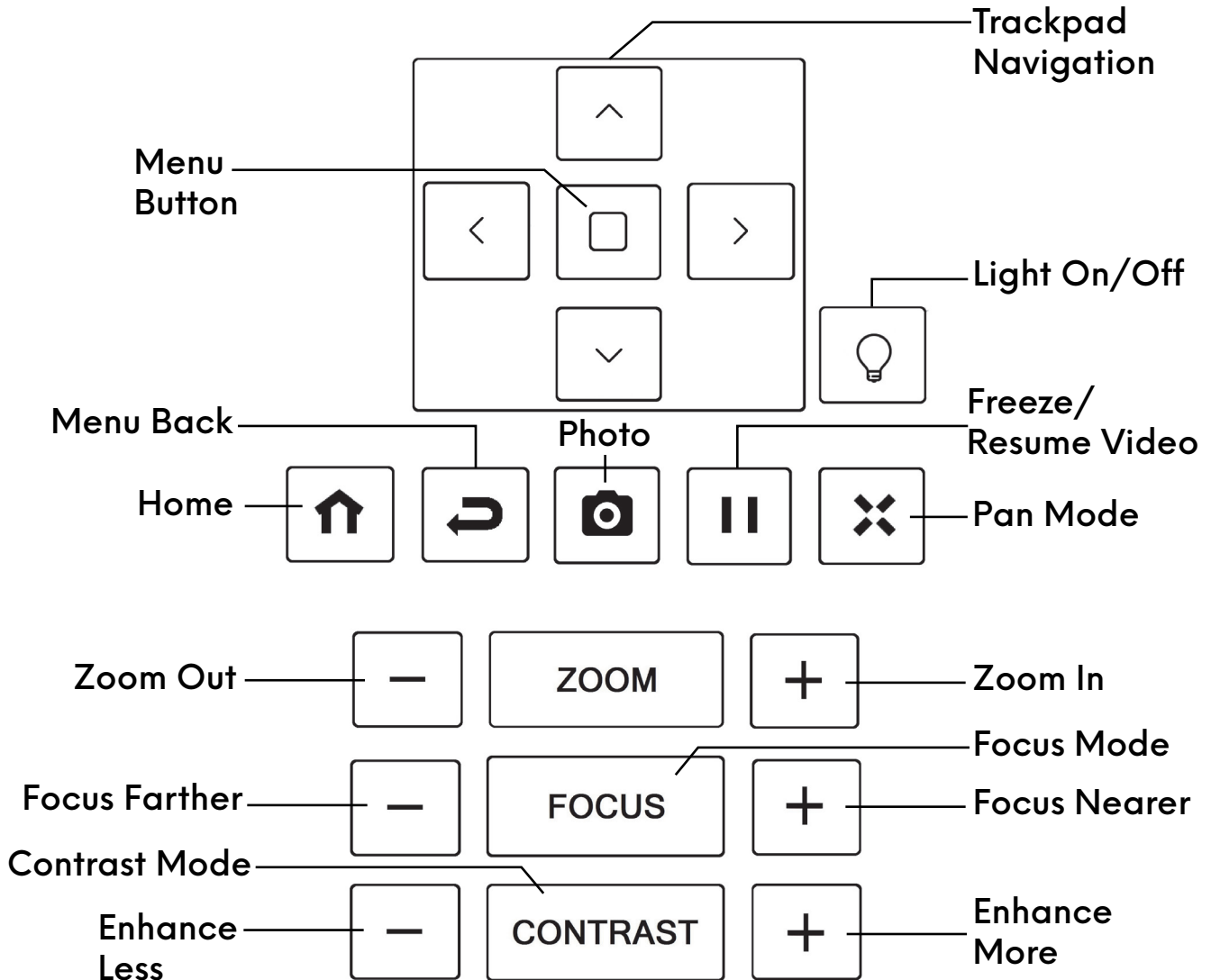


How to Use Your iPhone as a Remote Control for eSight 3 (Without Video)

1. Start eStreamer.
2. Connect your iPhone to your eSight 3.
3. In the eStreamer main menu, tap **Remote Control**.

How to Stop Receiving Video

- Tap the screen to show the status bar at the top of the screen, and then tap .

How to Resume Receiving Video

- With your iPhone connected to eSight 3, tap the screen to show the status bar at the top of the screen, and then tap .

How to Disconnect eStreamer from eSight 3

1. On your eSight 3, open the eSight menu.
2. Go to **SETUP > SYSTEM > CONNECT > ESTREAMER > DISCONNECT ALL**.

How to Block eStreamer Connections to eSight 3

1. On your eSight 3, open the eSight menu.
2. Go to **SETUP > SYSTEM > CONNECT > ESTREAMER > ALLOW** (No Checkmark).

"eSight 3" and "eSight" and associated images and symbols are trademarks of eSight Corporation. "eSight" is a registered trademark in Canada and the U.S. and may be pending or registered in other countries. Various U.S. and other patents pending. All other trademarks, registered trademarks, product names, and company names and/or logos cited herein, if any, are the property of their respective holders.

4000474 Rev. D © Copyright 2017, eSight Corporation.



1 Eglinton Ave. East Suite #401, Toronto, ON M4P 3A1
eSightEyewear.com • 1.855.837.4448 • VisionAdvocate@eSightEyewear.com