
Turn on eSight and the Remote

Turn on eSight. Wait for live video on the screens, and then turn on the remote.

Step 1 eSight On

5

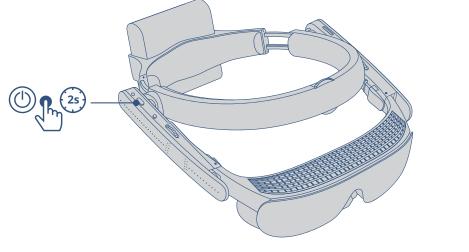

To continue, follow the onboarding on your eSight.


Connect & Register eSight


You'll need access to a computer or mobile phone with an internet or Wi-Fi connection to complete the Wi-Fi and eSight Account setup.

To generate your Wi-Fi QR Code and Setup your eSight Account Visit: eSightSetup.com or scan the QR Code with your phone to go to the website.





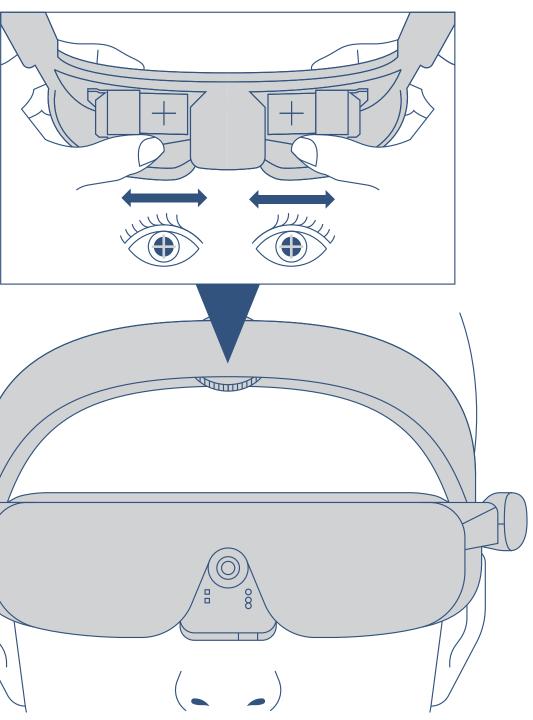

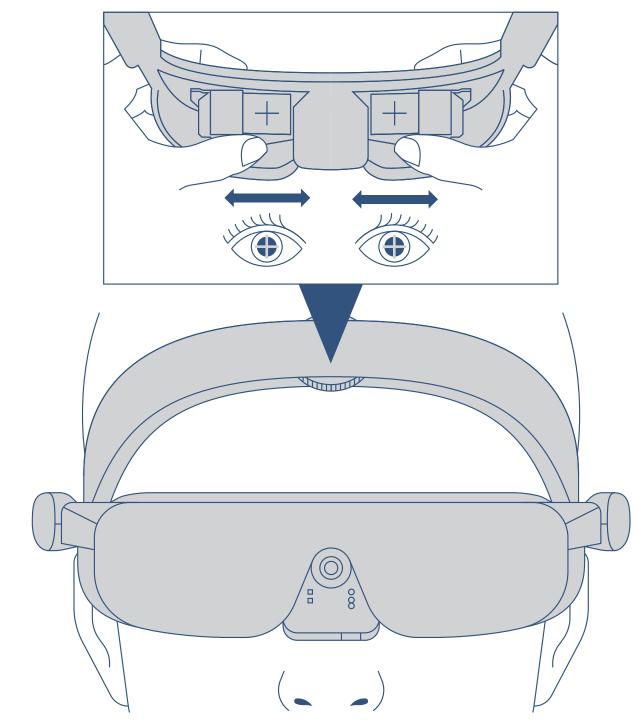

Step 2 Remote On


) Any Button

Set the Eye to Eye Distance

Slide the screens to center them over your eyes.

Set the Screen to Eye Distance

On both sides, rotate the **Arm Dial** forward to extend or backward to retract the visor.

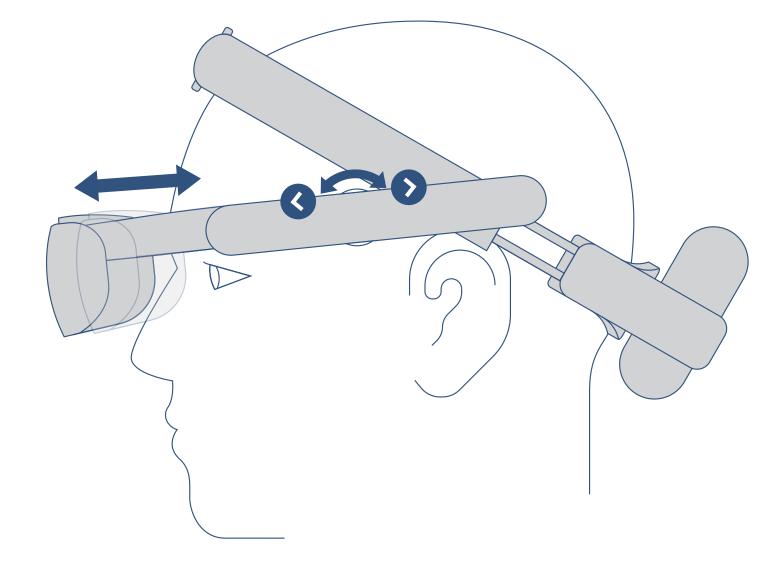

eSight(4)

Quick Start Guide

Introduction


This guide and the videos that display on your eSight will guide you through setting up your eSight and learning the controls.

The complete process takes 30-45 minutes and requires access to a computer or mobile phone with an internet or Wi-Fi connection. While you can do a partial setup in less than 10 minutes, completing the full process is recommended.


If you have a companion helping you set up eSight, they can view more detailed information about eSight at www.eSightEyewear.com/support

Let's begin!

4000834 Rev B © Copyright 2020 eSight Corporation All Rights Reserved. Publication Release May 2020

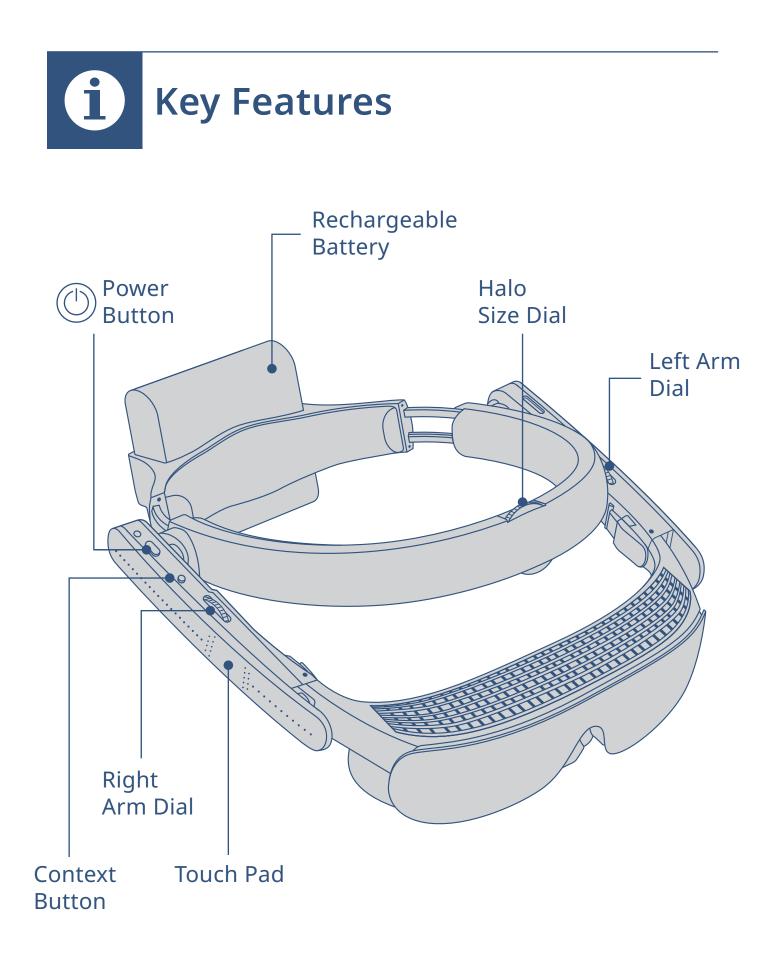


2

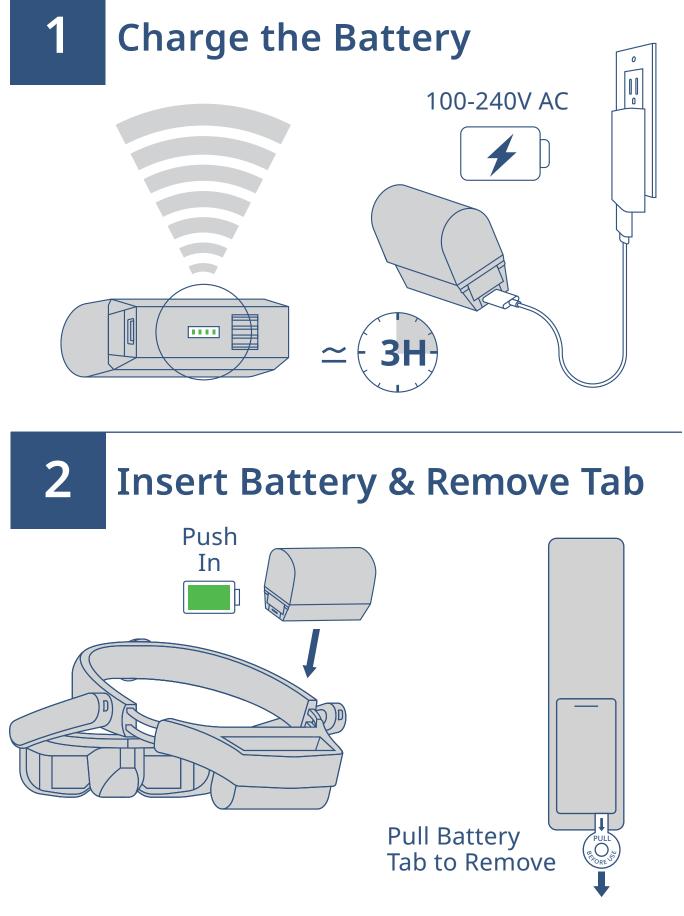



3

Secure & Center eSight

Secure eSight on your head and center eSight on your face.

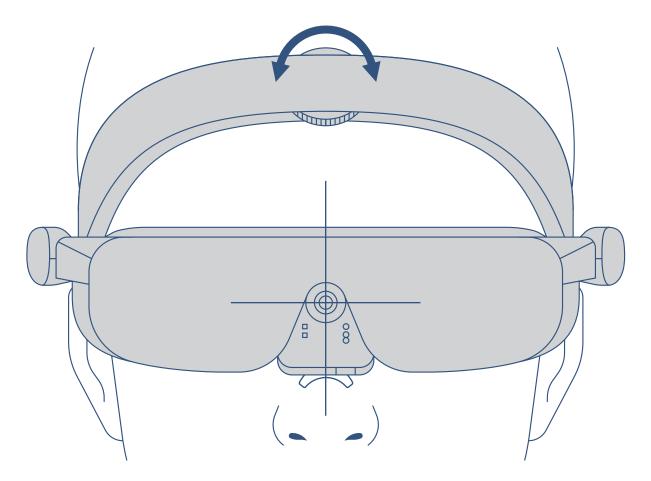

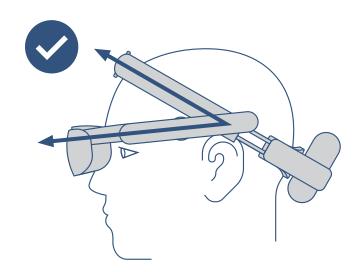

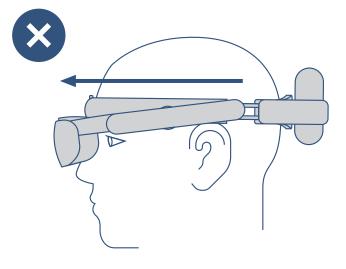

4

Adjust the Height of the Arms

Lower the arms to adjust the height of the visor so your eyes are vertically centered in the screens.



