

Reference Guide



eSight⁴

See new possibilities



eSight Corporation
515 Legget Drive, Suite 200
Ottawa, ON K2K 3G4
Canada

4000824 Rev. C © Copyright 2020, eSight Corporation. All rights reserved.

Publication Release: September 2020

eSight documentation is updated periodically for accuracy and completeness. Check [**esighteyewear.com/support**](https://esighteyewear.com/support) for the latest version.

No part of this publication may be reproduced without the express written permission of eSight Corporation.

“eSight Eyewear” and “eSight” and associated images and symbols are trademarks of eSight Corporation. “eSight” is a registered trademark in Canada and the U.S. and may be pending or registered in other countries. Various U.S. and other patents pending. All other trademarks, registered trademarks, product names, and company names and/or logos cited herein, if any, are the property of their respective holders.

eSight reserves the right to improve the contents of this document without notice.

About this Guide	1
Get to Know Gestures.....	3
eSight 4 Remote	5
Control eSight 4 via the Remote	7
Use the Basic Controls.....	10
Use the Advanced Controls.....	12
Navigate Menus, Home Screen, Gallery, Keyboard, & Message Boxes	17
Change Settings	18
Change the Appearance of Menu and Icons.....	18
Change Screen Settings.....	18
Adjust Sound Settings	19
Reset to System Defaults	20
Scan a Product Registration Code.....	20
Configure Modes	21
Take, View, and Manage Pictures and Video.....	24
Use the Gallery	24
Take and Manage Pictures.....	25
Record Video	27
Video Playback.....	27
Manage Videos	29
View HDMI Video.....	30
Connect an HDMI Video Source to My eSight.....	30

Connect an HDMI Video Source to My TV and eSight	31
Use the HDMI Controls.....	32
Manage Connections.....	34
Manage Wi-Fi Connections.....	34
Manage Bluetooth Connections.....	35
Use eReader.....	38
Get Help and Support	43
eSight App for Users	45
Control eSight 4 with eRemote	47
Use the Basic Controls.....	48
Use the Advanced Controls.....	50
Navigate Menus, Home Screen, Gallery, Keyboard, & Message Boxes	55
Manage Connections.....	56
Connect My Phone to eSight 4.....	56
Setup WiFi on eSight 4.....	57
Stream Out, Cast, or Mirror Content.....	58
Stream the eSight 4 Screen to the Phone Screen.....	58
Cast Media Content from the Phone to eSight 4.....	60
Mirror the Phone Screen to the eSight 4 Screen.....	62
Move Media from eSight 4 to the Phone.....	65
Use eReader.....	67

Manage Shared Access	71
Manage Profile	72
Account Sign Up, Sign In, and Recovery.....	74
eSight App for Caregivers	79
Stream the eSight 4 Screen to the Phone Screen	81
Account Sign Up for Caregivers	83
eSight Web for Users	85
eShare.....	87
Setup Wi-Fi	89
Manage Shared Access	91
Manage Profile	93
Account Sign Up, Sign In, and Recovery.....	96
eSight Web for Caregivers	101
eShare.....	103
Manage Profile	105
Account Sign Up, Sign In, and Recovery for Caregivers .	107

About this Guide

The Reference Guide covers how to work with eSight 4 using the remote control, the eSight App for mobile phones, and/or the eSight Web portal.

The Reference Guide is divided into the following key sections:

- **eSight Remote** — Learn how to control your eSight 4, access its features, and manage its settings using the remote.
- **eSight App for Users** — Learn how to use the eSight App to control eSight 4; setup Wi-Fi; share, cast, or mirror content; transfer pictures and videos; and manage caregiver access to your eSight 4.
- **eSight App for Caregivers** — Learn how to use the eSight App to create an eSight account and share the content of the eSight screen to your phone screen and control eSight 4.
- **eSight Web for Users** — Learn how to use eSight Web to share the your eSight screen and control eSight 4; setup Wi-Fi; and manage caregiver access to your eSight 4.

- **eSight Web for Caregivers** — Learn how to use eSight Web to create an eSight account and share the content of the eSight screen to your computer screen and control eSight 4.

Get to Know Gestures



Tap /Press




Tap for half a second / Press for half a second



Tap and Hold / Press and Hold



Release

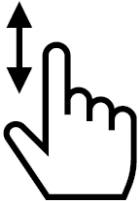

Swipe Up or Down




Swipe Left or Right




eSight 4 Remote

Control eSight 4 via the Remote


Basic Controls

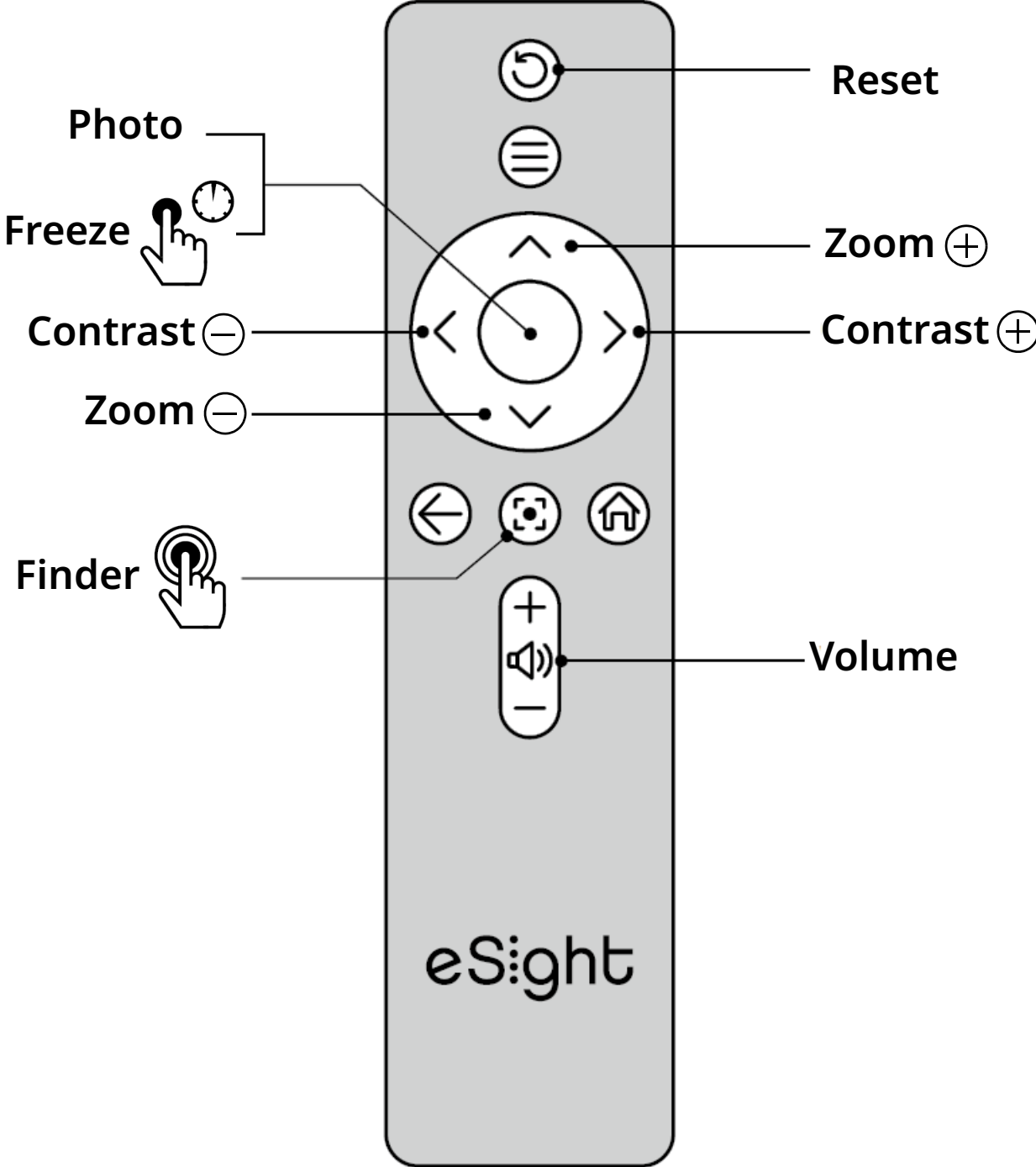

Advanced Controls

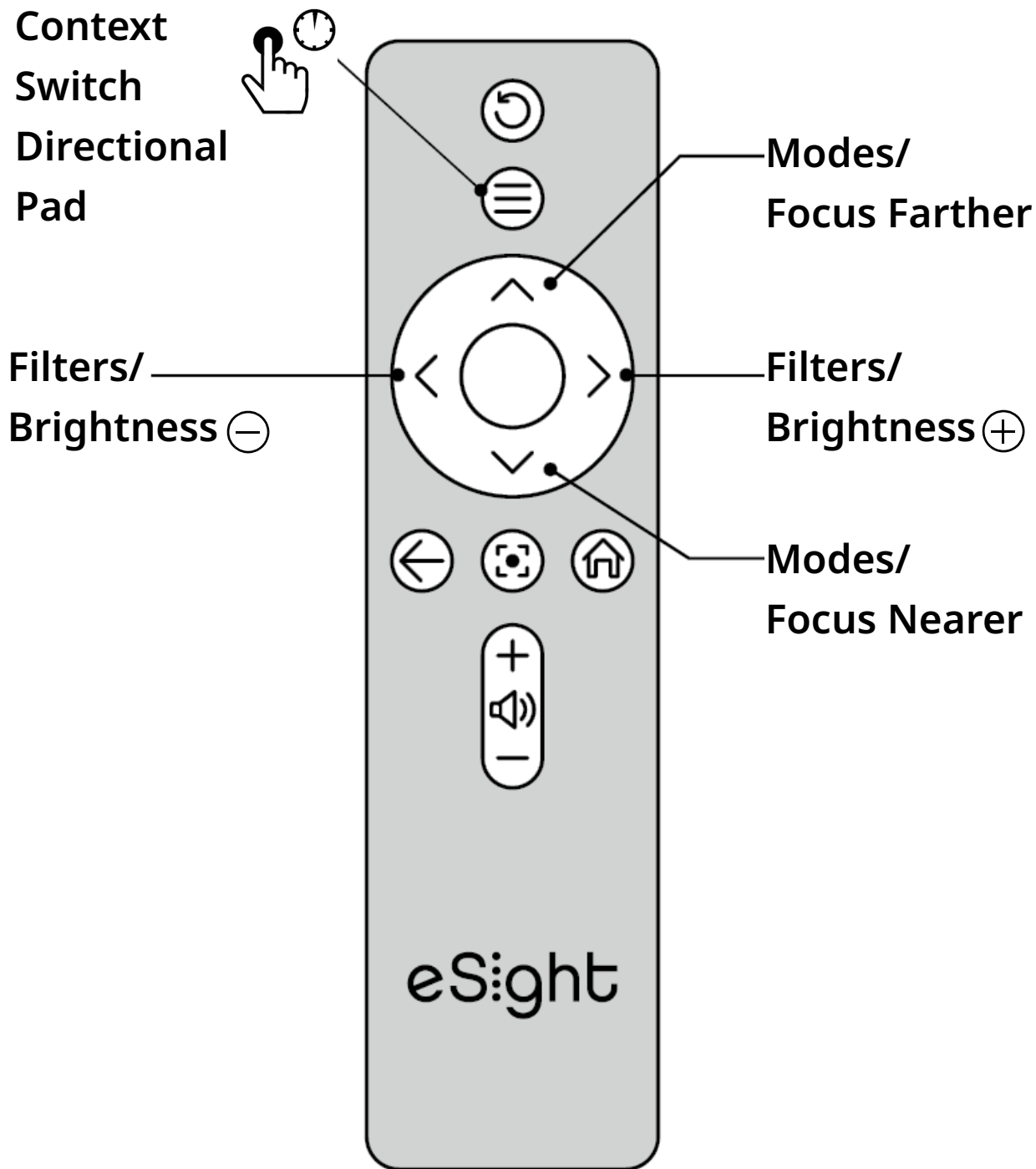


Context Menu and Home Screen Navigation

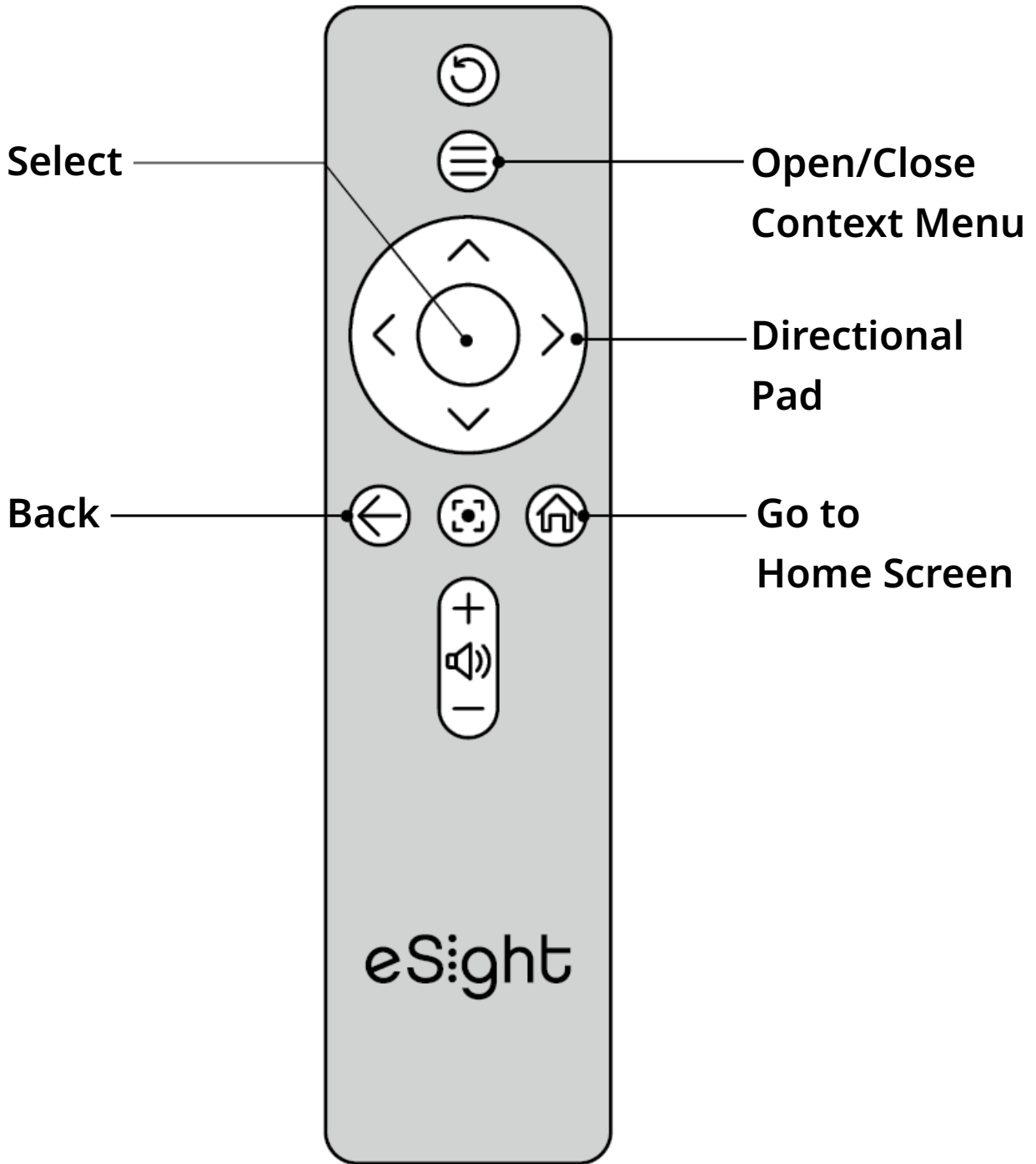


How do I...?

Use the Basic Controls


Zoom in / out

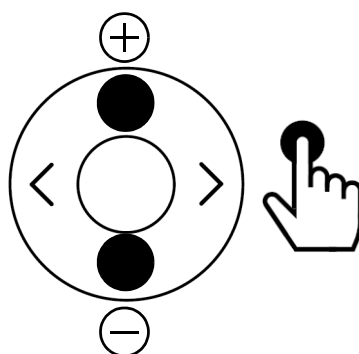

Adjust contrast

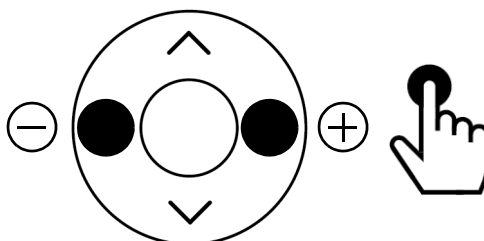


Take a picture

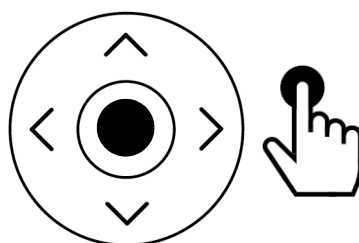

How do I...?

Freeze frame/Go back to live video

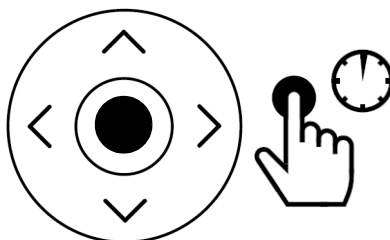

You can adjust zoom, contrast, and filters while in freeze frame.

Temporarily zoom out /Return to previous zoom level (Finder)




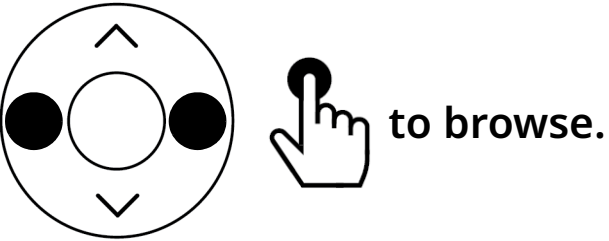
Reset view to default Mode settings



How do I...?

Use the Advanced Controls

Select a filter
(such as a
contrasting colour
scheme, like black
on white, white on
black, blue on
yellow,...)

- i** 
- ii**  to browse.
- iii** Stop to select.




D-Pad reverts to Zoom/Contrast if it is not used for a few seconds.

How do I...?

Apply a mode

A mode is a preset group of settings that can be applied at the same time for a specific activity or environment, like reading, TV/theatre, outdoors, or indoors.

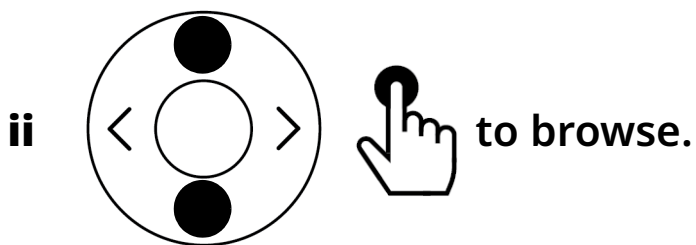

iii Stop to apply.




D-Pad reverts to Zoom/Contrast if it is not used for a few seconds.

How do I...?

Fine-tune the image brightness set by eSight (Image brightness compensation)



Use a Mode with Image Brightness set to Auto brightness.

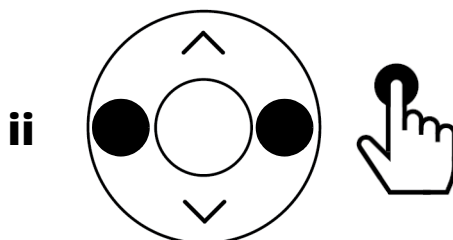


D-Pad reverts to Zoom/Contrast if it is not used for a few seconds.

How do I...?

Adjust and lock the image brightness for a specific environment (like a concert, movie theatre, etc.)



Use a Mode with Image brightness set to Manual brightness.

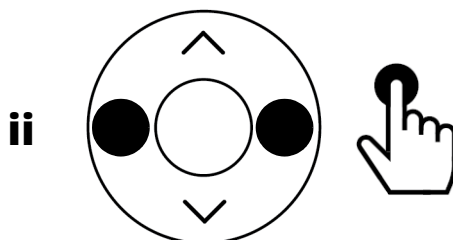

D-Pad reverts to Zoom/Contrast if it is not used for a few seconds.

How do I...?


Adjust the focus manually



Use a Mode with Focus set to Manual focus.



Focus Farther

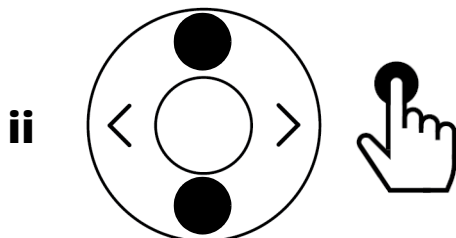

Focus Nearer




D-Pad reverts to Zoom/Contrast if it is not used for a few seconds.

Turn on/off the flashlight
(Live View only)



ii Go to **Flashlight**.

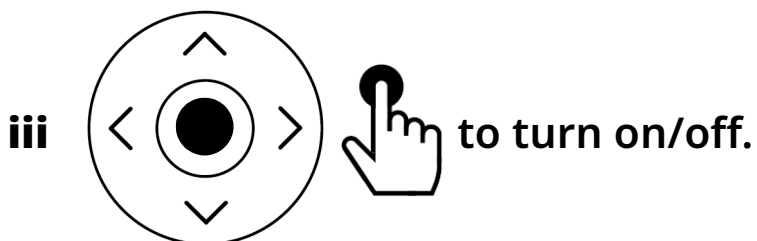

How do I...?

Navigate Menus, Home Screen, Gallery, Keyboard, & Message Boxes


Open/close the
Context Menu




Go to the Home
Screen



Browse

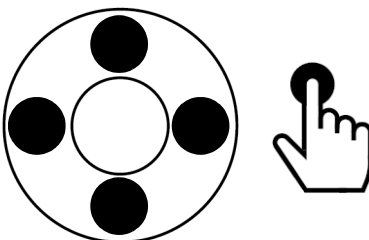

Select a
highlighted option

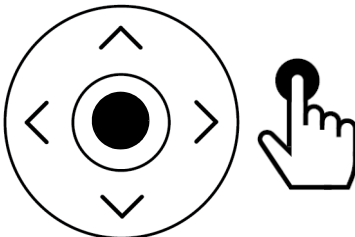

Go back or exit the
Home Screen



Change Settings

How do I...?

Change the Appearance of Menu and Icons

Change the colour

- Go to **Home Screen > Settings > Advanced > Appearance > Colour**, and then select your preferred option.


Change the size

- Go to **Home Screen > Settings > Advanced > Appearance > Size**, and then select your preferred option.

Change Screen Settings

Adjust the screen brightness

- i Go to **Home Screen > Settings > Screen Brightness**.

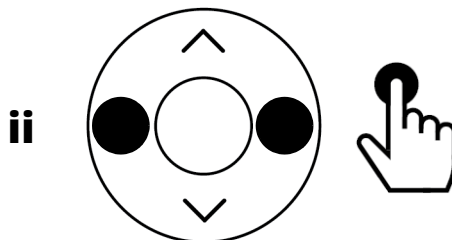

How do I...?

Select screens to use

(Both screens, Left screen only, or Right screen only)


- Go to **Home Screen > Settings > Advanced > Select screen**, and then select your preferred option.

Adjust Sound Settings

Set the system volume

(Notifications)


- i Go to **Home Screen > Settings > Sounds > System volume**.

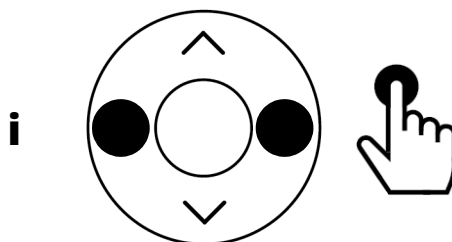

Set the media volume

(Videos, music, games, other media)


- i Go to **Home Screen > Settings > Sound > Media volume**.

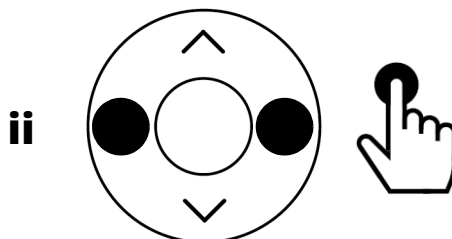

How do I...?

Set the narrator volume

- i Go to **Home Screen > Settings > Sound > Narrator volume.**

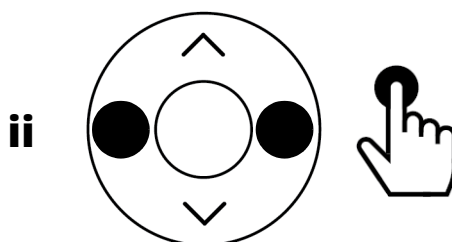

Reset to System Defaults

Restore eSight 4 to the initial device settings for Modes and Settings



Resetting does not delete your personal data, such as pictures, videos, registration,....

- i Go to **Home Screen > Settings > Advanced.**
- ii Select **Restore system defaults.**
- iii Select **OK** to confirm.

Scan a Product Registration Code

Scan an eSight 4 product registration code

- i Go to **Home Screen > Settings > Advanced.**
- ii Select **Setup eSight Account.**

Configure Modes

A mode is a preset group of settings that can be applied at the same time for a specific activity or environment, like reading, TV/theatre, outdoors, or indoors.

Customize each mode to your preferred settings. Later, set eSight to use the mode that most closely relates to your current activity or environment.

For example, you may like to control the focus and brightness manually while watching TV. You can customize the TV/theatre mode to use manual focus and manual brightness. When you're ready to watch TV, set eSight to TV/theatre mode and manually adjust the focus and brightness as necessary while watching TV.

When you're done watching TV, switch to a mode that has the best settings for your new activity.

How do I...?

Customize a mode

- i Go to **Home Screen** > **Settings** > **Configure mode** > { Mode Name }.**
- ii Go to **Image brightness**:**
 - **Manual brightness** — Select if you want to manually set and lock the image brightness to a preferred level for an activity (like watching concerts or movies with fast changing lighting).
 - **Auto brightness** — Select if you want eSight to adjust the brightness.
- iii Go to **Focus**:**
 - **Manual Focus** — Set if you need to lock the focus at a preferred distance to prevent unwanted refocusing (while watching concerts or movies or reading).
 - **Auto Focus** — Select if you want eSight to adjust the focus.

Continues on next page

How do I...?

Customize a
mode (Continued)

iv Go to Filters:

- Go to {Filter Name} (Default), and then select the filter that you want applied when you use the mode.
 - In main filter list, select the filters that you want access to when you use the mode.
-

Take, View, and Manage Pictures and Video

How do I...?

Use the Gallery


Open the gallery • Go to **Home Screen > Gallery**.

Open a picture or video • In the Gallery, browse to the picture or video, and then select.



You can adjust zoom, contrast, and filters (pictures) or contrast (videos).

Go to next or previous picture / video

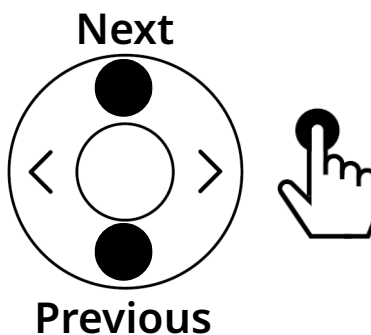

How do I...?

Delete a picture or video from the gallery

i In Gallery, highlight the picture or video.



iii Select **Delete selected**.

iv Select **OK**.

Delete everything from the gallery




ii Select **Delete all**.

iii Select **OK**.

Take and Manage Pictures

Take a picture (Live View only)

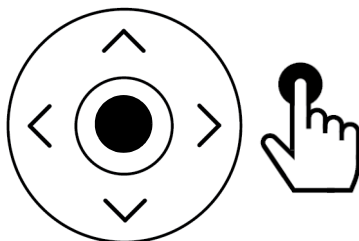


Open a picture

i Go to **Home Screen > Gallery**.


ii In the Gallery, browse to the picture, and then select.

How do I...?

Pan the picture I am viewing

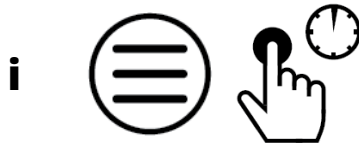

Zoom In

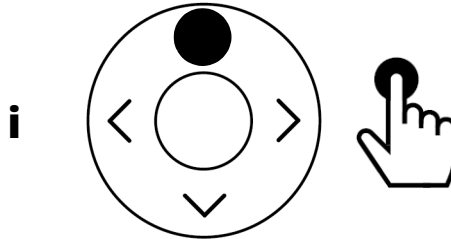

ii Move your head to pan.

Delete the picture I am viewing



ii Select **Delete photo**.

iii Select **OK**.


How do I...?

Record Video

Record a video
(Live View only)




ii Select **Record video**.

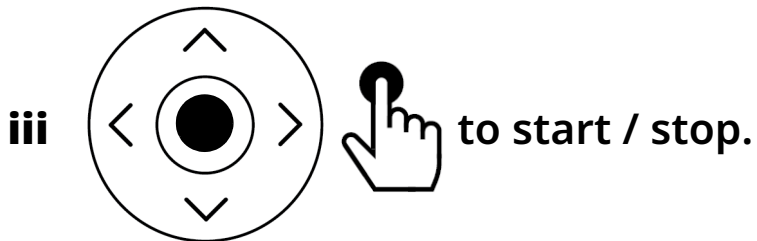

You can adjust zoom and contrast while recording.

Exit to Live View

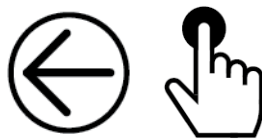

Video Playback


Open a video

i Go to **Home Screen > Gallery**.


ii In the **Gallery**, browse to the video, and then select.

How do I...?

Play/Pause a video

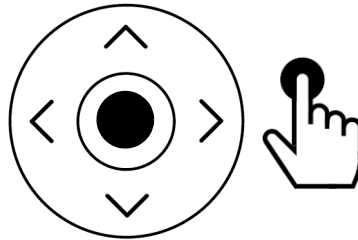

Skip forward or backward in a video

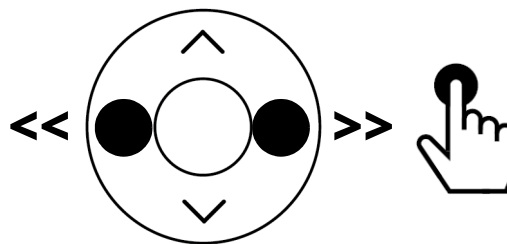

Push a second (4x), third (8x), or fourth time (x2) to change the speed.

Play a video I am viewing from the beginning



ii Select **Play from beginning**.

How do I...?

Manage Videos

Delete the video I
am viewing

i




ii Select **Delete video**.

iii Select **OK**.

View HDMI Video

How do I...?

Connect an HDMI Video Source to My eSight

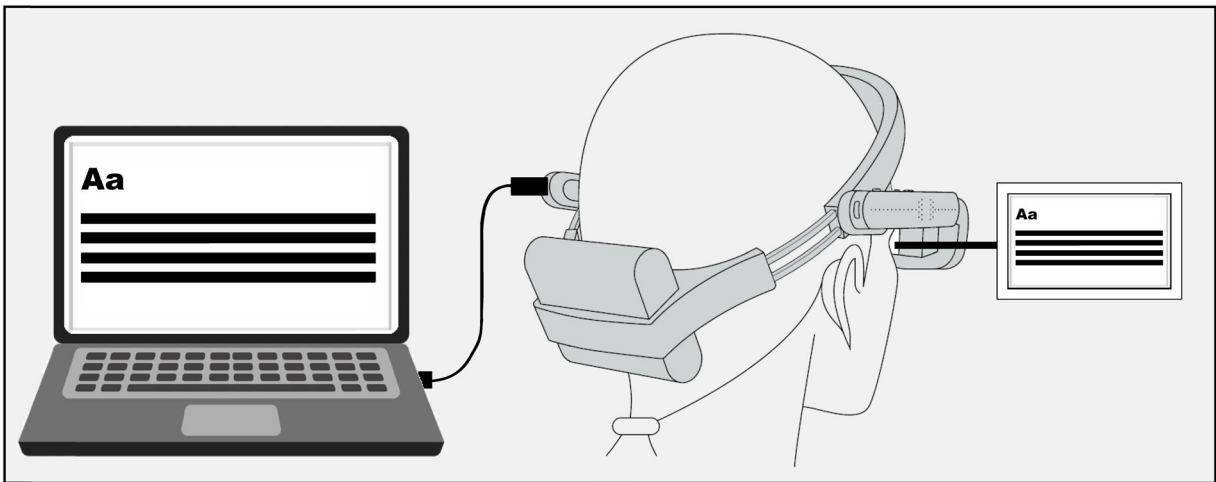


- 1** Plug the large end of the eSight 4 HDMI Cable into HDMI Out on the external video source. For example, a laptop.
- 2** Plug the small end into the back of the left arm on eSight 4.

The HDMI video displays in your eSight 4 when it detects a supported video signal.

See “Specifications” in the eSight 4 User Guide for the list of supported resolutions.

How do I...?

Connect an HDMI Video Source to My TV and eSight

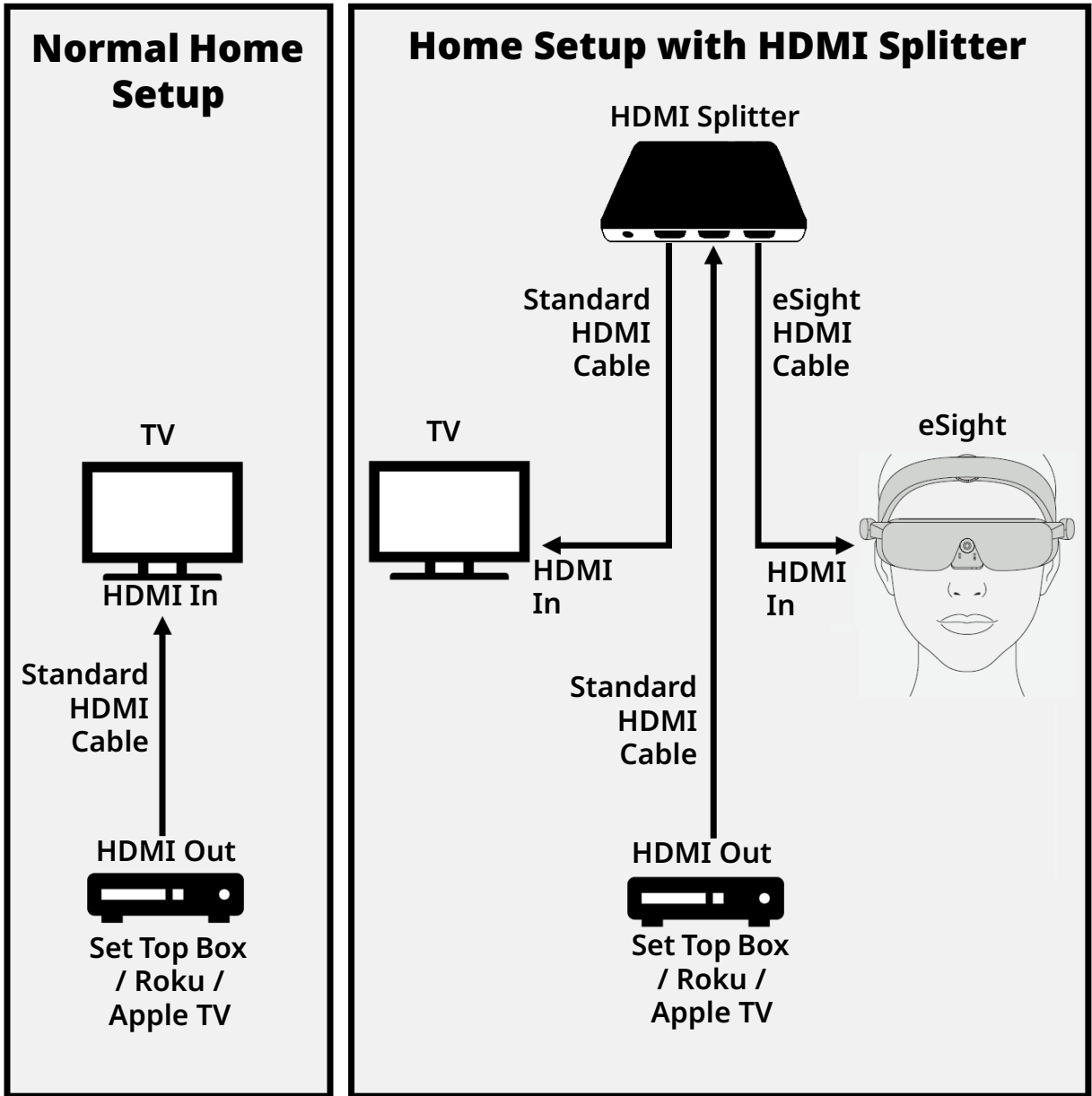


How do I...?

The eSight-approved splitter is an optional accessory that allows you to connect an HDMI video source to the TV and eSight 4.


Consult the documentation accompanying the splitter for setup instructions.

Use the HDMI Controls


Zoom in / out

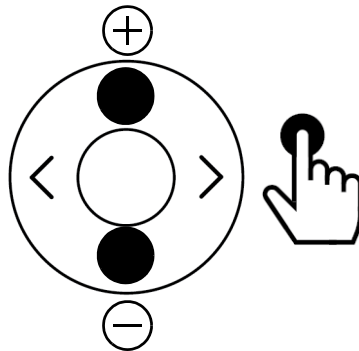

Adjust contrast

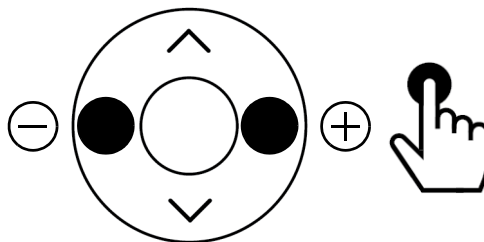


Pan the HDMI video

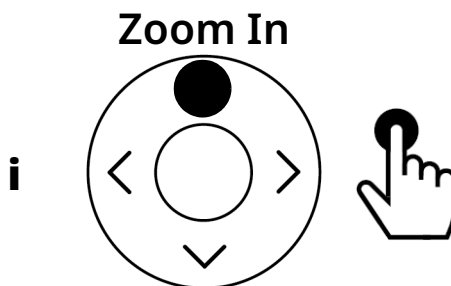

ii Move your head to pan.

How do I...?

Apply a filter to the HDMI video

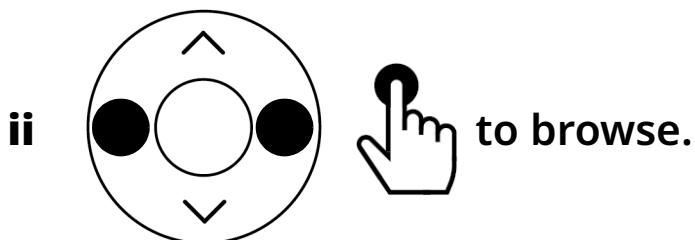

iii Stop to select.



D-Pad reverts to Zoom/Contrast if it is not used for a few seconds.

Reset to default view



Manage Connections

How do I...?

Manage Wi-Fi Connections

Search for available networks

- i** Go to **Home Screen** > **Settings** > **Wi-Fi**.
- ii** Select **Search**.

Scan a Wi-Fi QR code to connect to an available network



Use the eSight app to create a Wi-Fi QR Code. See “Generate a Wi-Fi QR Code” on page 57.

- i** Display the generated Wi-Fi code on your device.
- ii** Go to **Home Screen** > **Settings** > **Wi-Fi** > **QR scanner**.
- iii** Position the on-screen box on the QR code.



If you don't have a remote, setup Wi-Fi from the eSight 4. Press the **Context button**, and then select **Setup Wi-Fi**.

How do I...?

Connect to a saved Wi-Fi network

- i** Go to **Home Screen** > **Settings** > **Wi-Fi**.
- ii** Select the name of the saved network in the Available networks list to which you want to connect, and then select **Connect**.

Forget a saved Wi-Fi network

- i** Go to **Home Screen** > **Settings** > **Wi-Fi**.
- ii** Select the name of the saved network that you want to forget.
- iii** Select **Forget**.
- iv** Select **Forget** to confirm.

Manage Bluetooth Connections

Pair to an eSight Bluetooth remote

- i** On the remote, push the **Reset button**.
- ii** On eSight 4, press the **Context button**.

Continues on next page

How do I...?

Pair to an eSight Bluetooth remote (Continued)

- iii On the touch pad, swipe to highlight **Bluetooth pairing**, and then tap to select.
- iv From the list, swipe to highlight **eSight**, and then tap to select.

Pair and connect a supported Bluetooth device (like headphones, speakers, or mobile phones) to eSight

- i Put your Bluetooth device into “discovery” mode.
- ii Go to **Home Screen > Settings > Bluetooth**.
- iii From the **New devices** list, select the name of the device that you want to pair to eSight, and then select **Pair**.



If you don't see your Bluetooth device listed, select **Search** to refresh the list.

How do I...?

Reconnect a paired Bluetooth device

- i** Go to **Home Screen** > **Settings** > **Bluetooth**.
- ii** From the **New devices** list, select the name of the paired device that you want to connect to eSight, and then select **Connect**.

Disconnect a Bluetooth device

- i** Go to **Home Screen** > **Settings** > **Bluetooth**.
- ii** From the **Paired devices** list, select the name of the device that you want to disconnect from eSight, and then select **Disconnect**.

Forget a Bluetooth device

- i** Go to **Home Screen** > **Settings** > **Bluetooth**.
 - ii** From the **Paired devices** list, select the name of the device that you want to unpair from eSight, and then select **Unpair**.
-

Use eReader

How do I...?

Add a PDF document to eSight to view in eReader



If you are transferring files between your Mac computer and eSight 4, you must first install the Android File Transfer app on your Mac computer. This application is required so you can browse and transfer files between devices.

Go to **<https://www.android.com/filetransfer/>** and follow the accompanying instructions.

- i Using the USB power cable, connect your eSight 4 to your computer or laptop.

Continues on next page

How do I...?

Add a PDF document to eSight to view in eReader
(Continued)

- ii On your computer, open the file manager.
The eyewear shows up in the file manager as eSight 4.
- iii Copy the document that you want to read to the following folder on eSight 4: **eSight Eyewear G4\Internal shared storage\Documents**


Open a document in eReader

- i Go to **Home Screen > eReader**.
- ii Select the document you want to read.




You can adjust zoom, contrast, and filters while viewing a document.

View the previous or next page in an open document

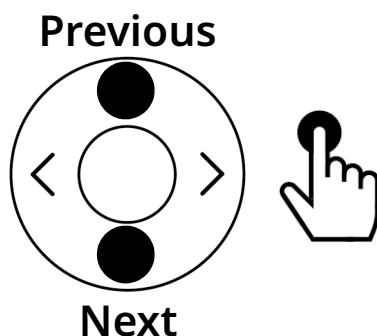

How do I...?


Zoom in/Zoom Out

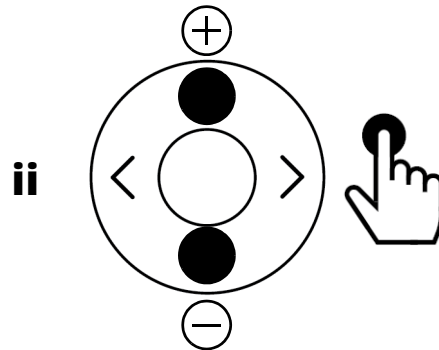

Directional Pad reverts to Page Movement if it is not used for a few seconds.


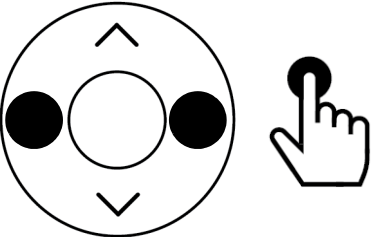
Pan a page

Zoom in, wait until the Directional Pad reverts to Page Movement, and then

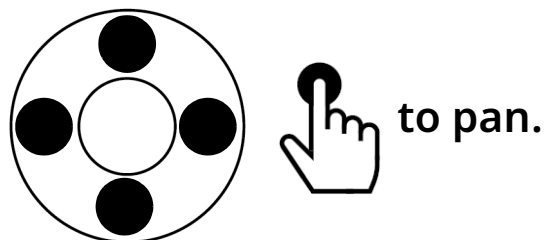

How do I...?

Apply a filter to the document

- i  x 2
- ii  to browse.
- iii Stop to select.



Directional Pad reverts to Page Movement if it is not used for a few seconds.

Reset to default page view



Delete the open document

- i 
 - ii Select **Delete file.**
 - iii Select **Delete.**
-

How do I...?

Go back to the
document list
or previous
view



Get Help and Support

How do I...?	
View product documentation	<ul style="list-style-type: none">• Go to Home Screen > Support > Help documents.
View the on-boarding tutorial from the beginning	<ul style="list-style-type: none">• Go to Home Screen > Support > Onboarding > Start from beginning.
Resume the tutorial	<ul style="list-style-type: none">• Go to Home Screen > Help and support > Onboarding > Resume.
Adjust eye to eye distance	<ul style="list-style-type: none">i Go to Home Screen > Help and support > Onboarding > Adjust eye to eye distance.ii Follow the on-screen instructions.

How do I...?

Adjust screen to eye distance

- i Go to **Home Screen > Help and support > Onboarding > Adjust screen to eye distance.**
- ii Follow the on-screen instructions.

View notifications

- i Go to **Home Screen > Help and support > Notifications.**
- ii Select the notification you want to view.

Update the software on my eSight 4

- **Home Screen > Help and support > Notifications > Update.**


View device information, like product serial number, software version number, storage information, regulatory certifications

- **Go to Home Screen > Help and support > Device information.**
-



eSight App for Users

Control eSight 4 with eRemote

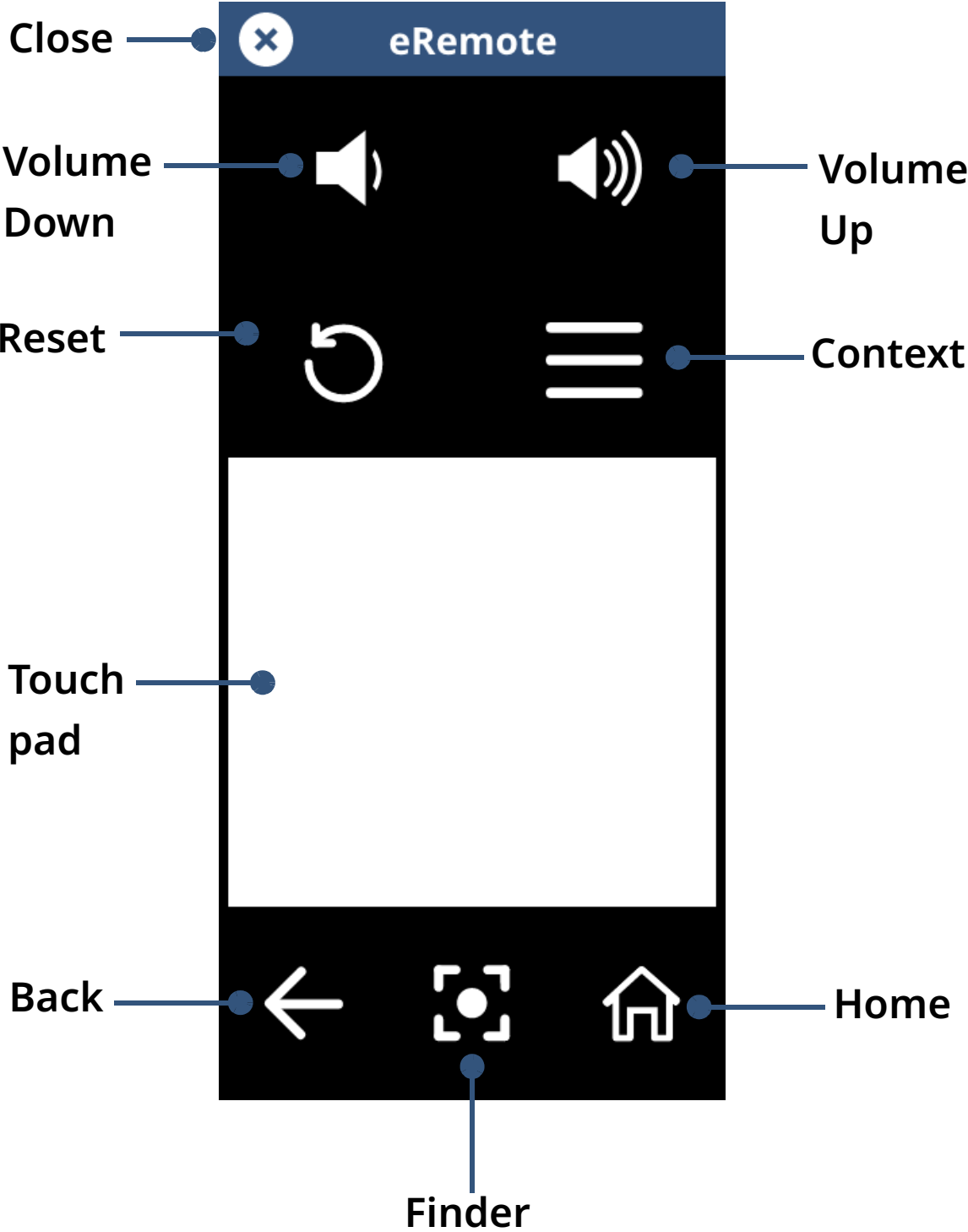


How do I...?

Use the Basic Controls


Open eRemote




eSight and your phone must be paired via Bluetooth to use eRemote. See “Connect My Phone to eSight 4” on page 56.

At the bottom of the eSight App Home screen, tap eRemote .


Zoom in / out

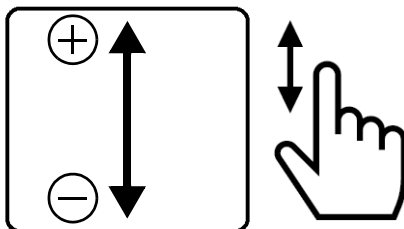

Adjust contrast

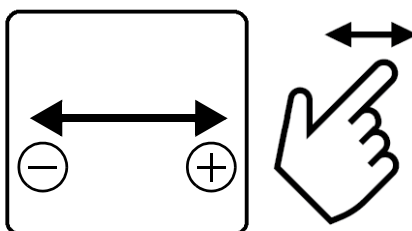


Take a picture

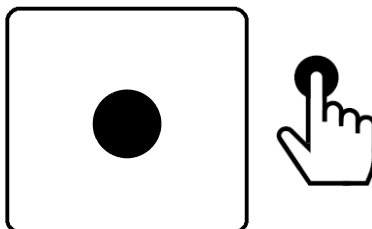

How do I...?

Freeze frame/Go back to live video

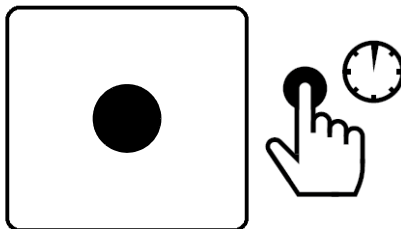

You can adjust zoom, contrast, and filters while in freeze frame.


Pan an image while in Freeze frame

- Zoom in, then move your head to pan.

Reset view to default Mode settings



Temporarily zoom out /Return to previous zoom level (Finder)

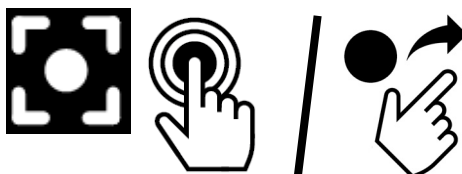



Close eRemote



How do I...?

Use the Advanced Controls

Select a filter
(such as a
contrasting colour
scheme, like black
on white, white on
black, blue on
yellow,...)

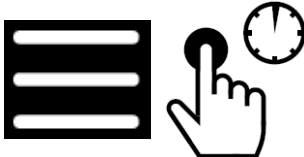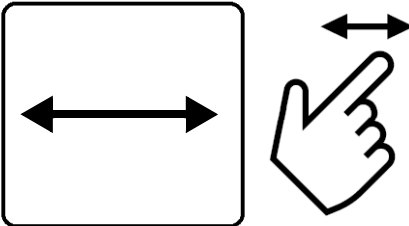- i 
- ii  to browse.
- iii Stop to select.





D-Pad reverts to Zoom/Contrast if it is not used for a few seconds.

How do I...?

Apply a mode

A mode is a group of settings tuned for a specific activity, like reading, TV/theatre, outdoors, indoors...

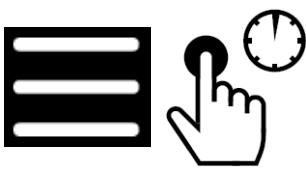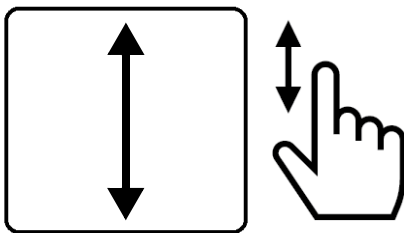- i** 
- ii**  to browse.
- iii** Stop to apply.



D-Pad reverts to Zoom/Contrast if it is not used for a few seconds.


How do I...?

Fine-tune the image brightness set by eSight (Image brightness compensation)




Use a Mode with Image Brightness set to Auto brightness.

i

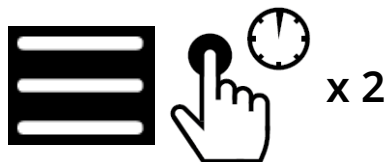

ii

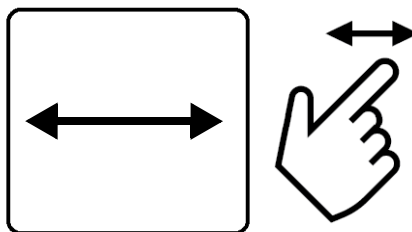

D-Pad reverts to Zoom/Contrast if it is not used for a few seconds.


How do I...?

Adjust and lock the image brightness for a specific environment (like a concert, movie theatre, etc.)




Use a Mode with Image brightness set to Manual brightness.

i

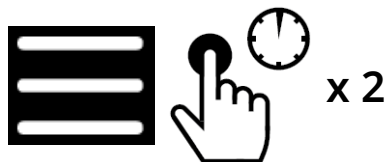

ii

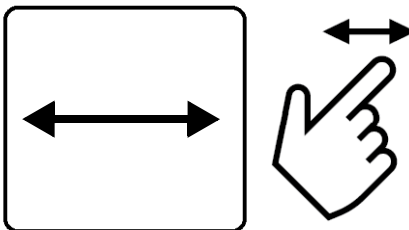


D-Pad reverts to Zoom/Contrast if it is not used for a few seconds.

How do I...?


Adjust the focus manually



Use a Mode with Focus set to Manual focus.

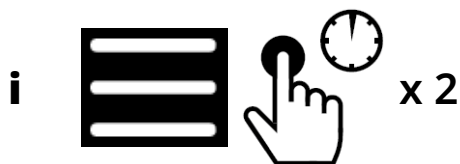

Focus Farther

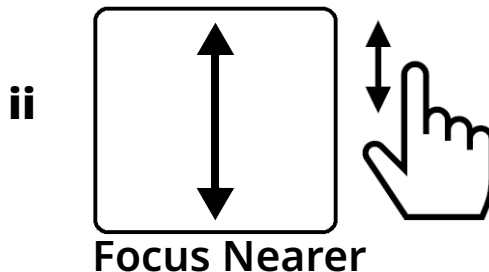

Focus Nearer




D-Pad reverts to Zoom/Contrast if it is not used for a few seconds.

Turn on/off the flashlight
(Live View only)



i Go to **Flashlight**.

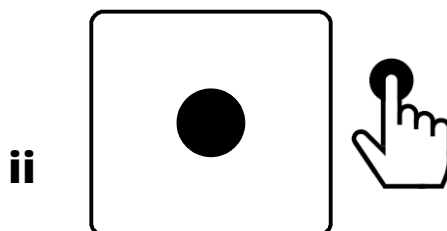

How do I...?

Navigate Menus, Home Screen, Gallery, Keyboard, & Message Boxes


Open/close the Context Menu




Go to the Home Screen



Browse

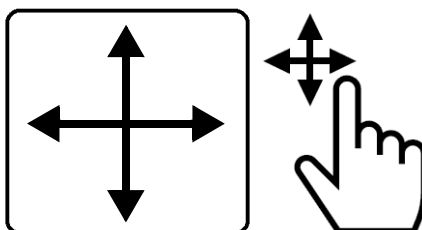

Select a highlighted option

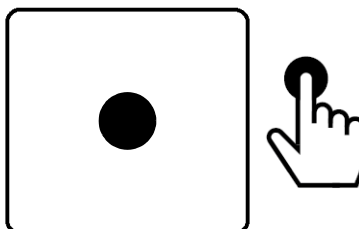

Go back or exit the Home Screen



Manage Connections

How do I...?

Connect My Phone to eSight 4

Connect my phone to eSight 4



Bluetooth must be enabled on your phone to connect your phone to eSight 4.

- i** Go to **My eSight > Connectivity > Pair to an eSight.**

If Bluetooth is turned off on your phone, you are prompted to turn on your phone's Bluetooth. Tap **Turn on Bluetooth** to continue.

- ii** From the **Available Devices** list, tap **eSight 4.**

eSight 4 appears in the **Paired devices** list when your phone successfully connects to the eSight.

- iii** Tap **Back.**
-

How do I...?

Setup WiFi on eSight 4

Generate a
Wi-Fi QR Code

- i** Go to **My eSight > Connectivity > Setup Wi-Fi on eSight**.
- ii** In the **Wi-Fi name** box, type the network name (also called the SSID).
- iii** In the **Wi-Fi password** box, type the network password.
- iv** From the **Wi-Fi network type** list, select the type of security (WEP, WPA, WPA2 or no encryption) used to secure the network.
- v** Tap **Create Wi-Fi Code**.

Scan a Wi-Fi QR
code to
connect to an
available
network

- i** Display the generated Wi-Fi code on your phone.
 - ii** On the right arm of eSight, press the **Context button**, and then swipe the touch pad to highlight **Setup Wi-Fi**, and then tap to select.
 - iii** Position the on-screen box on the QR code.
-

Stream Out, Cast, or Mirror Content



Streaming, casting, or mirroring requires that

- Wi-Fi and Bluetooth are enabled on your phone.
- eSight and your phone are paired via Bluetooth. See “Connect My Phone to eSight 4” on page 56.
- eSight and your phone are connected to the same Wi-Fi network. See “Setup WiFi on eSight 4” on page 57.
- eSight is turned on and within range of the Wi-Fi network.

Stream the eSight 4 Screen to the Phone Screen

eShare allows you to send the content of the eSight screen to your phone screen.

How do I...?

Start stream out

- Go to **My eSight > eShare**.
-

How do I...?

Stop stream out



ii Tap **Stop sharing**.



You can stop screen sharing from the eSight 4. On the right arm of eSight, press the **Context button**, and then select **Stop sharing**.

Use the digital remote

- Use the digital remote to control eSight 4. It mimics the controls of the physical remote. See “Control eSight 4 via the Remote” on page 7.
-

Cast Media Content from the Phone to eSight 4

eCast allows you to stream video and photos from your phone to your eSight 4.

How do I...?

Cast media from
Android

- i** On the eSight App Home screen, tap **eCast**.
- ii** From the displayed media on your phone, tap the media content that you want to display on eSight, and then tap **View on eSight**.

Cast Media from
iOS




You can only cast media that is saved on your phone. You cannot cast media that is stored online.

- i** On the eSight App Home screen, tap **eCast**.
 - ii** From the gallery on your phone, select the media content that you want to display on eSight, and then tap **View on eSight**.
-


How do I...?

Use the eCast Controls

Go to the previous or next media

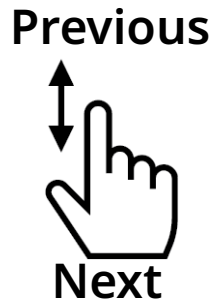

Rewind or fast forward

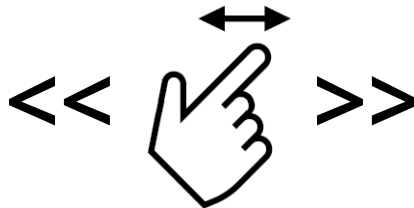

Play/Pause



Stop casting

- On the eSight App, tap **Stop casting**.



You can stop casting from the eSight 4. On the right arm of eSight, press the **Context button**, and then select **Exit Casting**.

Mirror the Phone Screen to the eSight 4 Screen

Screen mirroring allows you to view content from your phone screen on your eSight 4.



How do I...?

Mirror the Phone Screen to eSight 4 Screen

Mirror from
Android


- On the eSight App Home screen, tap **eMirror**, and then tap **Start now**.

Mirror from iOS
11

- i On the eSight App Home screen, tap **eMirror**.
 - ii In iOS, go to **Settings > Control Center > Customize Controls** then tap  next to **Screen Recording**.
 - iii Swipe up from the bottom edge of any screen.
 - iv Press deeply on the grey icon  and tap **Microphone**.
 - v Tap **Start Recording**, then wait for the three-second countdown.
-


How do I...?

Mirror from
iOS12+


- i On the eSight App Home screen, tap **eMirror**.
- ii Tap the Mirroring button , and then select the eSight app.
- iii Tap **Start Broadcast**.

Stop Mirroring


Stop mirroring
from Android

- On the eSight App Home screen, tap the **Stop**  button in the bottom right corner of your screen.

Stop mirroring
from iOS 11

- Open Control Center and tap the red icon , or
- Tap the red status bar at the top of your screen, and tap **Stop**.

Stop mirroring
from iOS 12

- Tap the Mirroring button , or
 - Tap the red status bar at the top of your screen, and tap **Stop**.
-

How do I...?

Stop mirroring
from the remote



ii **Select Disconnect.**

iii **Select Yes.**

Stop mirroring
from eSight 4

i On the right arm of eSight, press
the **Context button**, and then
select **Disconnect**.

ii **Select Yes.**

Move Media from eSight 4 to the Phone



Moving media from eSight 4 to your phone requires that

- Wi-Fi and Bluetooth are enabled on your phone.
- eSight and your phone are paired via Bluetooth. See “Connect My Phone to eSight 4” on page 56.
- eSight and your phone are connected to the same Wi-Fi network. See “Setup WiFi on eSight 4” on page 57.
- eSight is turned on and within range of the Wi-Fi network.
- Sufficient storage space is available on your phone for the media transfer.

How do I...?

Move videos or pictures from eSight 4 to my phone

i On the eSight App Home screen, tap **Gallery**.

Continues on next page

How do I...?

Move videos or pictures from eSight 4 to my phone
(Continued)

- ii** Tap to select the video(s) and/or picture(s) that you want to transfer to your phone or tap **Select All**.
 - iii** Tap **Transfer**.
 - iv** Once the transfer is complete, tap **Close**.
-

Use eReader

How do I...?

Add a PDF document to eSight to view in eReader



If you are transferring files between your Mac computer and eSight 4, you must first install the Android File Transfer app on your Mac computer. This application is required so you can browse and transfer files between devices.

Go to **<https://www.android.com/filetransfer/>** and follow the accompanying instructions.

- i Using the USB power cable, connect your eSight 4 to your computer or laptop.

Continues on next page

How do I...?

Add a PDF document to eSight to view in eReader
(Continued)

- ii On your computer, open the file manager.
The eyewear shows up in the file manager as eSight 4.
- iii Copy the document that you want to read to the following folder on eSight 4: **eSight 4\Internal shared storage\Documents**


Open a document in eReader

- i Go to **eRemote > Home Screen > eReader**.
- ii Select the document you want to read.




You can adjust zoom, contrast, and filters while viewing a document.

View the previous or next page in an open document

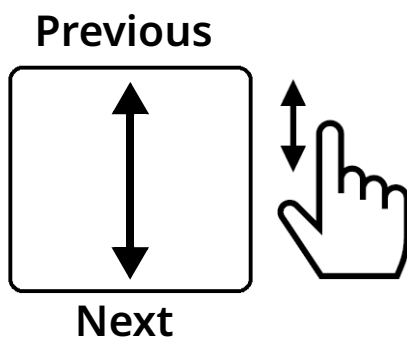

How do I...?


Zoom in/Zoom Out

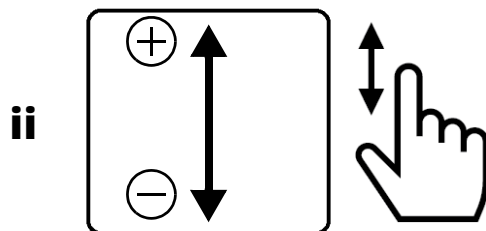

Directional Pad reverts to Page Movement if it is not used for a few seconds.





Pan a page

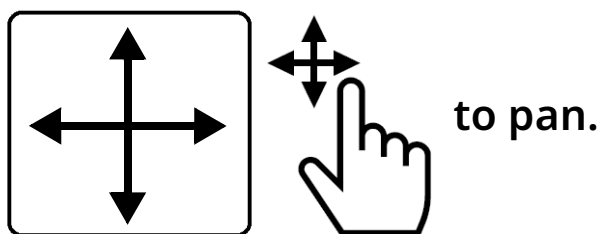Zoom in, wait until the Directional Pad reverts to Page Movement, and then



How do I...?


Apply a filter to the document

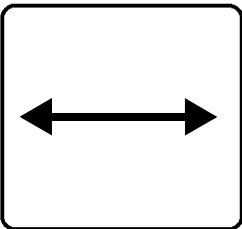- i   x 2
- ii   to browse.
- iii Stop to select.



Directional Pad reverts to Page Movement if it is not used for a few seconds.


Reset to default page view

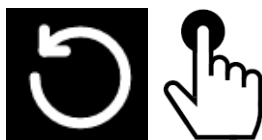

Delete the open document

- i  
- ii Select **Delete file**.

Go back to the document list or previous view

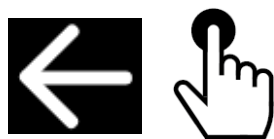

Manage Shared Access

How do I...?

Grant access to my eSight

- i** Go to **My eSight > Shared Access**.
 - ii** Tap **Invite a new user**.
 - iii** Type the email address and name of the invitee.
 - iv** Tap **Send invitation**.
 - v** Tap **Send invitation** to confirm.
-

Remove access to my eSight

- i** Go to **My eSight > Shared Access**.
 - ii** From the **Members** or **Invited members** list, tap the person for whom you want to remove access.
 - iii** Tap **Remove access**.
 - iv** Tap **Remove access** to confirm.
-

Resend an invitation

- v** Go to **My eSight > Shared Access**.
 - vi** From the **Invited members** list, tap the person to whom you want to resend an invitation.
 - vii** Tap **Resend invitation**.
 - viii** Tap **Close**.
-

Manage Profile



Your phone must be connected to a network to update your profile settings.

How do I...?

Update my personal information in my account profile

- i** On the eSight App Home screen, tap your profile photo, and then tap **Manage Account**.
- ii** Tap the personal information you want to update, and make the required change.
- iii** Tap **Save**.

Add or update my profile photo

- i** On the eSight App Home screen, tap your profile photo, and then tap **Manage Account**.
 - ii** Tap **Change photo**.
 - iii** Tap **Take a photo** to use the phone camera or tap **Upload photo** to select where you'd like to import your picture from.
-

How do I...?

- | | |
|----------------------------------|---|
| Change my account password | <ul style="list-style-type: none">i On the eSight App Home screen, tap your profile photo, and then tap Change password.i Type your current password.ii Type a new password in the New password and Confirm new password boxes.iii Tap Continue.iv Tap Close. |
| View shared access notifications | <ul style="list-style-type: none">• On the eSight App Home screen, tap your profile photo, and then tap Notifications. |
| Manage accessibility settings | <ul style="list-style-type: none">i On the eSight App Home screen, tap your profile photo.ii Tap App settings > Device settings.iii Tap Device settings. |
| Access help documents | <ul style="list-style-type: none">• On the eSight App Home screen, tap your profile photo, and then tap Help. |

Account Sign Up, Sign In, and Recovery



Your phone must be connected to a network to create or sign in to your eSight account.

How do I...?

Create my eSight account

- i** Open the eSight app.
- ii** Tap **Create an account**.
- iii** In the **Email** box, type the email address to associate with your eSight account, and then tap **Continue**.
- iv** In the **Password** and **Confirm password** boxes, type the password you want to use to sign in to your eSight account, and then tap **Continue**.
eSight sends a confirmation email to the email address you specified in step iii.

Continues on next page

How do I...?

Create my eSight account
(Continued)

You have 24 hours to confirm your email address.

- v** In your email software, open the confirmation email, and tap **Confirm email** to verify your email address.
- vi** In the eSight app, tap **Go to Sign in**.
- vii** Type your email address and password, and then tap **Sign in**.



Bluetooth must be enabled on your phone to connect your phone to eSight.

- viii** Follow the on-screen instructions to complete your account profile, register your eSight, and pair your phone to your eSight.
-

How do I...?

Sign in to my eSight account

- i** Open the eSight app.
 - ii** Tap **Sign in to eSight**.
 - iii** Enter your eSight account email and password.
 - iv** Tap **Sign in**.
-

Recover my eSight account password

- i** Open the eSight app.
- ii** Tap **Sign in to eSight**.
- iii** Tap **Forgot your password?**.
- iv** In the **Email address** box, type the email address associated with your eSight account.
- v** Tap **Send**.

eSight sends an email with steps to follow to recover your eSight account. You have one hour to reset your password.

- vi** In your email application, open the email from eSight, and tap **Reset your password**.

Continues on next page

How do I...?

Recover my eSight
account password
(Continued)

vii In the **Reset password** screen, type a new password in the **Password** and **Confirm password** boxes.

viii Tap **Reset password**.

ix On the **Sign in** screen, type your email address and new password.



eSight App for Caregivers

Stream the eSight 4 Screen to the Phone Screen

eShare allows you to send the content of the eSight screen to your phone screen.



Streaming requires that

- Wi-Fi and Bluetooth are enabled on your phone.
- eSight and your phone are paired via Bluetooth. See “Connect My Phone to eSight 4” on page 56.
- eSight and your phone are connected to the same Wi-Fi network. See “Setup WiFi on eSight 4” on page 57.
- eSight is turned on and within range of the Wi-Fi network.

How do I...?

Start eShare

- i On the eSight App Home screen, tap the name of user associated with the eSight 4 you want to access.

Continues on next page


How do I...?

Start eShare (Continued)

- ii** Tap **eShare**.
- iii** Wait for the user to approve your request for access.
When you connect successfully, you see what the user sees.
- iv** Use the digital remote to control eSight 4. It mimics the controls of the physical remote. See “Control eSight 4 via the Remote” on page 7.

Stop eShare

- i** In the top left corner,

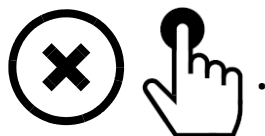

- ii** Tap **Disconnect** to confirm.

Use the digital remote

- Use the digital remote to control eSight 4. It mimics the controls of the physical remote. See “Control eSight 4 via the Remote” on page 7.
-

Account Sign Up for Caregivers



Your phone must be connected to the internet to sign up for an eSight app account.

How do I...?

Sign up for an eSight app account

- i** In your email software, open the invitation email, and follow the on-screen instructions to confirm your email address and download the eSight app.
- ii** Open the eSight app.
- iii** In the **Email address** box, type the email address to which the invitation was sent.
- iv** In the **Password** box, type the temporary password from the invitation.
- v** Tap **Sign In**.

Continues on next page

How do I...?

Sign up for an
eSight app account
(Continued)

vi In the **Password** and **Confirm password** boxes, type a new password.

vii Tap **Save**.

viii Follow the on-screen instructions to complete your account profile or tap **Skip for now** to go to the Home Screen.



eSight Web for Users

eShare



eShare requires that eSight is turned on and connected to the internet.

eShare allows you to send the content of your eSight screen to your computer screen. From your computer, you can use the on-screen digital remote to control eSight 4.

How do I...?

Start eShare

- i** On the Home page, click **eShare**.
- ii** On the eShare page, click **Connect**.

Stop eShare

- iii** On the eShare page, click **Disconnect**.



You can stop screen sharing from the eSight 4. On the right arm of eSight, press the **Context button**, and then select **Stop sharing**.

How do I...?

Use the digital remote

- **Use the digital remote to control eSight 4. It mimics the controls of the physical remote. See “Control eSight 4 via the Remote” on page 7.**
-

Setup Wi-Fi

How do I...?

Generate a
Wi-Fi QR Code

- i** On the Home page, click **Setup Wi-Fi**.
 - ii** In the **Wi-Fi name** box, type the network name (also called the SSID).
 - iii** In the **Wi-Fi password** box, type the network password.
 - iv** From the **Network type** list, select the type of security (WEP, WPA, WPA2 or no encryption) used to secure the network.
 - v** Click **Generate Wi-Fi Code**.
-

How do I...?

- Scan a Wi-Fi QR code to connect to an available network
- i** Display the generated Wi-Fi code on your computer.
See “Generate a Wi-Fi QR Code” on page 89.
 - ii** On the right arm of eSight, press the **Context button**, and then swipe the touch pad to highlight **Setup Wi-Fi**, and then tap to select.
 - iii** Position the on-screen box on the QR code.
-

Manage Shared Access

How do I...?

Grant access to my eSight

- i** On the Home page, click **Shared Access**.
 - ii** On the **Current Members** tab, click **Invite new member**.
 - iii** Type the email address and name of the invitee.
 - iv** Click **Send invite**, and then click **Confirm**.
-

Remove access to my eSight

- i** On the Home page, click **Shared Access**.
 - ii** From the **Current members** list, click **Remove** beside the name of the person for whom you want to remove access.
 - iii** Click **Confirm**.
-

How do I...?

Resend an invitation

- i** On the Home page, click **Shared Access**.
 - ii** From the **Pending members** list, click **Edit** beside the name of the person to whom you want to resend the invitation.
 - iii** Click **Resend invitation**.
-

Revoke an invitation

- i** On the Home page, click **Shared Access**.
 - ii** From the **Pending members** list, click **Edit** beside the name of the person for whom you want to cancel the invitation.
 - iii** Click **Revoke invitation**.
-

Manage Profile

How do I...?

Update my personal information in my account profile

- i** On the top right-hand side of the page, click your name, and then click **My account**.
- ii** On the **Manage profile** page, click **Edit** beside the personal information you want to update, and make the required change(s).
- iii** Click **Save**.

Add or update my profile photo

- i** On the top right-hand side of the page, click your name, and then click **My account**.
 - ii** Click **Edit photo**.
 - iii** Select where you'd like to import your picture from.
 - iv** Click **Save**.
-

How do I...?

- Change my account password
- i** On the top right-hand side of the page, click your name, and then click **My account**.
 - ii** In the left pane, click **Change password**.
 - iii** In the **Old password** box, type your current password.
 - iv** Type a new password in the **New password** and **Re-enter new password** boxes.
 - v** Click **Confirm change password**.
-

- View shared access notifications
- i** On the top right-hand side of the page, click your name, and then click **My account**.
 - ii** In the left pane, click **Notifications**.
-

- Access help documents
- i** On the top right-hand side of the page, click your name, and then click **My account**.
 - ii** In the left pane, click **Help**.
-

How do I...?

Sign Out

- On the top right-hand side of the page, click your name, and then click **Sign out**.
-

Account Sign Up, Sign In, and Recovery



Your computer must be connected to the internet to create or sign in to your eSight account.

How do I...?

Create my eSight account

- i** Using your Internet browser, go to **eSightSetup.com**.
- ii** Click the link to create or sign in to an eSight account.
- iii** Click **Register new device**.
- iv** In the **Email** box, type the email address to associate with your eSight account.
- v** In the **Password** and **Re-enter password** boxes, type the password you want to use to sign in to your eSight account.
- vi** Click **Sign Up**.

Continues on next page

How do I...?

Create my eSight account
(Continued)

eSight sends a confirmation email to the email address you specified in step iv.

You have 24 hours to confirm your email address.

vii In your email software, open the confirmation email, and click **Activate** to verify your email address.

viii On the **Account is Active** page, click **Sign In**.

ix Type your email address and password, and then click **Sign In**.

x Follow the on-screen instructions to complete your account profile and register your eSight.

How do I...?

- Sign in to my eSight account
- i** Using your Internet browser, go to **eSightSetup.com**.
 - ii** Click the link to create or sign in to an eSight account.
 - iii** Click **Sign In**.
 - iv** Enter your eSight account email and password.
 - v** Click **Sign In**.
-

- Recover my eSight account password
- i** Using your Internet browser, go to **eSightSetup.com**.
 - ii** Click the link to create or sign in to an eSight account.
 - iii** Click **Sign In**.
 - iv** Click **Forgot password?**
 - v** In the **Email** box, type the email address associated with your eSight account.

Continues on next page

How do I...?

Recover my eSight account password (Continued)

- vi Click **Send**.**
eSight sends an email with steps to follow to recover your eSight account. You have one hour to reset your password.
 - vii In your email application, open the email from eSight, and click **Reset password**.**
 - viii In the **Reset password** screen, type a new password in the **New password** and **Re-enter new password** boxes.**
 - ix Click **Confirm**.**
 - x Click **Sign In**.**
On the **Sign In** screen, type your email address and new password.
-



**eSight Web
for Caregivers**

eShare



eShare requires that eSight is turned on and connected to the internet.

eShare allows you to send the content of the eSight screen to your computer screen. From your computer, you can use the on-screen digital remote to control eSight 4.

How do I...?

Start eShare

- i** On the Home page, click **Connect** beside the name of user associated with the eSight 4 you want to access.
 - ii** Wait for the user to approve your request for access.
When you connect successfully, you see what the user sees.
 - 3** Use the digital remote to control eSight 4. It mimics the controls of the physical remote. See “Control eSight 4 via the Remote” on page 7.
-

How do I...?

Stop eShare

- On the eShare page, click **Disconnect**.
-

Use the digital remote

- Use the digital remote to control eSight 4. It mimics the controls of the physical remote. See “Control eSight 4 via the Remote” on page 7.
-

Manage Profile

How do I...?

Update my personal information in my account profile

- i** On the top right-hand side of the page, click your name, and then click **My account**.
- ii** On the **Manage profile** page, click **Edit** beside the personal information you want to update, and make the required change(s).
- iii** Click **Save**.

Change my account password

- i** On the top right-hand side of the page, click your name, and then click **My account**.
- ii** In the left pane, click **Change password**.
- iii** In the **Old password** box, type your current password.

Continues on next page

How do I...?

Change my account password
(Continued)

- iv Type a new password in the **New password** and **Re-enter new password** boxes.
 - v Click **Confirm change password**.
-

Sign Out

- On the top right-hand side of the page, click your name, and then click **Sign out**.
-

Account Sign Up, Sign In, and Recovery for Caregivers



Your computer must be connected to the internet to sign up for an eSight account.

How do I...?

Sign up for an eSight account

- i** In your email software, open the invitation email.
- ii** Click **Sign In using web**.
- iii** In the **Email** box, type the email address to which the invitation was sent.
- iv** In the **Password** box, type the temporary password from the invitation.
- v** Click **Sign In**.
- vi** In the **New password** and **Re-enter new password** boxes, type a new password.

Continues on next page

How do I...?

Sign up for an eSight account
(Continued)

vii Click **Confirm**.

viii Follow the on-screen instructions to complete your account profile or click **Skip and complete profile later** to go to the Home Screen.

Sign in to my eSight account

i Using your Internet browser, go to **eSightSetup.com**.

ii Click the link to create or sign in to an eSight account.

iii Click **Sign In**.

iv Enter your eSight account email and password.

v Click **Sign In**.

How do I...?

Recover my eSight account password

- i** Using your Internet browser, go to **eSightSetup.com**.
- ii** Click the link to create or sign in to an eSight account.
- iii** Click **Sign In**.
- iv** Click **Forgot password?**
- v** In the **Email** box, type the email address associated with your eSight account.
- vi** Click **Send**.

eSight sends an email with steps to follow to recover your eSight account. You have one hour to reset your password.
- vii** In your email application, open the email from eSight, and click **Reset password**.

Continues on next page

How do I...?

Recover my eSight
account password
(Continued)

- viii** In the **Reset password** screen, type a new password in the **New password** and **Re-enter new password** boxes.
 - ix** Click **Confirm**.
 - x** Click **Sign In**.
 - xi** On the **Sign In** screen, type your email address and new password.
-