



## How to Prepare for a Demo

1. Make sure at least one External Battery is charged. You can shake these batteries to light up the indicator LEDs on the bottom.
2. If the device hasn't been used for over one week, charge the internal battery by plugging a fully charged external battery into the headset 30-60 minutes before the device needs to be used.
  - There is an internal battery that powers the device and allows the user to switch external batteries without powering the unit down. If this internal battery gets low, the device will start to act strange and may turn off immediately after start up.
3. Turn on the headset and make sure the remote connects when you press any button. Change the batteries if necessary.
4. Review controls for magnification and any other settings that may come up during the demo.
5. Gather any materials you might need, such as eye charts or paired eSight App devices.